


New Pneumatic Back Tilt

Salon Top & Tilt Top Tables




Pneumatic tables come equipped with a non-electric pneumatic assist mechanism. The following steps will guide you through operating the pneumatic back tilt control and adjusting the back to the desired level.

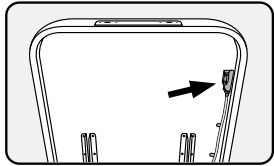


Step 1

Locate the pneumatic back tilt handle, which is located on the right side under the back tilt.

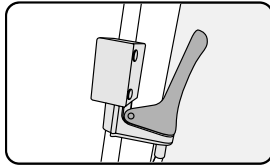


Step 2

The handle is set to the locking position.

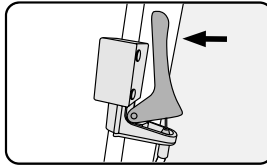

Step 3

Squeeze/press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

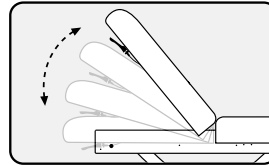


Step 4

With the handle squeezed/pressed, move the back tilt to the desired level and release the handle. It will lock in place.

New Pneumatic Leg Break


Salon Top, Leg Break

Tables with the pneumatic option come equipped with the same non-electric, pneumatic assist mechanisms as mentioned above. The following steps will guide you through operating the pneumatic leg break control.

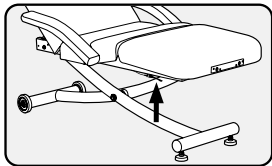


Step 1

Locate the pneumatic leg break control, which is located under the front section of the table.

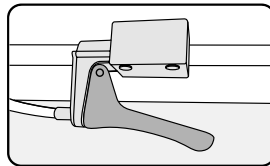


Step 2

The handle is set to the locking position.

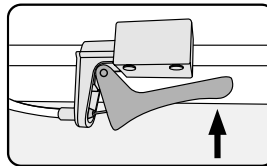

Step 3

Squeeze/press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

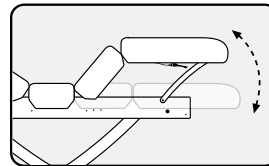

Step 4

With the handle squeezed/pressed, move the back tilt to the desired level and release the handle. It will lock in place.