


## New Pneumatic Back Tilt

### Salon Top & Tilt Top Tables




Pneumatic tables come equipped with a non-electric pneumatic assist mechanism. The following steps will guide you through operating the pneumatic back tilt control and adjusting the back to the desired level.

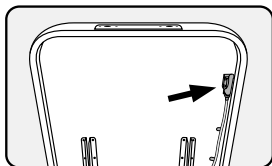


#### Step 1

Locate the pneumatic back tilt handle, which is located on the right side under the back tilt.

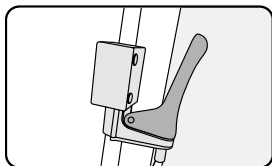


#### Step 2

The handle is set to the locking position.

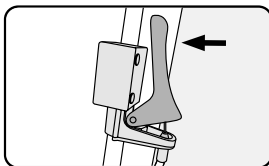

#### Step 3

Squeeze/press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

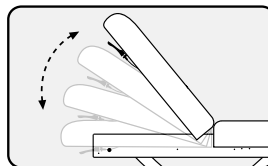


#### Step 4

With the handle squeezed/pressed, move the back tilt to the desired level and release the handle. It will lock in place.

## New Pneumatic Leg Break


### Salon Top, Leg Break

Tables with the pneumatic option come equipped with the same non-electric, pneumatic assist mechanisms as mentioned above. The following steps will guide you through operating the pneumatic leg break control.

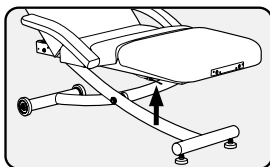


#### Step 1

Locate the pneumatic leg break control, which is located under the front section of the table.

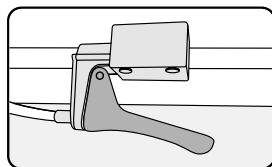


#### Step 2

The handle is set to the locking position.

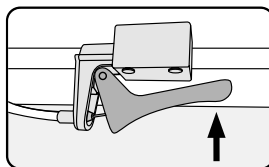

#### Step 3

Squeeze/press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

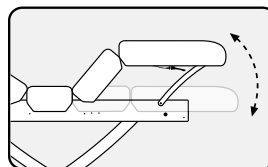

#### Step 4

With the handle squeezed/pressed, move the back tilt to the desired level and release the handle. It will lock in place.