



**Everest Lift Table**



**Everest**



**Everest Spa**



**Everest Eclipse**

*\*tables shown with optional accessories and/or finishes*

© Copyright 2020  
Earthlite®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Earthlite.

Earthlite® is a registered trademark of Earthlite, LLC.

#### Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Earthlite®.

Earthlite® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Earthlite® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Earthlite® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

# Introduction

Congratulations on your purchase of the multi-functional Everest lift table. The Everest has been designed for maximum comfort and to meet the specific positioning needs of your clients. The elegant single column design of the Everest and Everest Spa allows for plenty of clearance underneath the table for leg and knee room, while still providing exceptional strength. The Everest Eclipse adds a beautiful hardwood cabinet, providing a more custom and upscale look.

Everest tables are designed for licensed therapists and are intended to be used in a professional spa, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The Everest table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 599-1112 for international customers. We can also be reached via e-mail at [info@earthlite.com](mailto:info@earthlite.com).

## Table of Contents

### Section 1: Safety Information

Safety Symbols .....	2
Warnings .....	2-3

### Section 2: Product Overview

Product Photos .....	4
Product Specifications .....	4

### Section 3: Installation

Moving Your Table .....	5
Basic Setup .....	5
Leveling Instructions .....	5

### Section 4: Operating Instructions

Weight/Lift Capacities .....	6
Electrical Specifications .....	7
Key Lockout .....	7
Hand/Foot Control .....	8
Pneumatic Adjustments .....	9
Manual Adjustments .....	10
LED Chroma Lighting .....	11

### Section 5: Care & Maintenance

Care Instructions .....	12
Troubleshooting .....	13
Warranty Information .....	13
Contact Us .....	back cover

# SECTION 1: SAFETY INFORMATION

## Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

**IMPORTANT: The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.**



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

## IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.




### RISK OF ELECTRIC SHOCK, FIRE, AND EXPLOSION

- 1) Always unplug this furnishing from the electrical outlet before servicing.
- 2) Never use this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- 3) Keep the cord away from heated surfaces.
- 4) Do not use outdoors.
- 5) Do not operate where aerosol (spray) projects are being used or where oxygen is being administered.
- 6) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 7) Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.

## GROUNDING INSTRUCTIONS:

**POWER SUPPLY:** (120V AC grounded) Electrical limitations: Do not use extension cord with this product, plug directly into grounded outlet. **This product must be grounded.** If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. This three prong grounded plug will only fit into wall outlet one way. If the plug does not fit fully in the outlet, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

### NORTH AMERICA ONLY

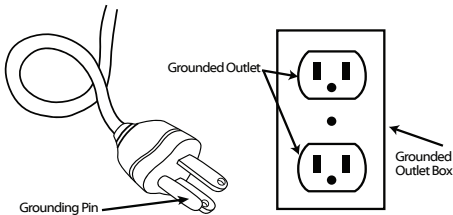

### INTERNATIONAL VERSIONS ONLY

This product is for use on a nominal 220V-240V circuit and has a grounding plug (plug type varies by country). Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

# SECTION 1: SAFETY INFORMATION

## WARNING




**To reduce the risk of burns, fire, electric shock, or injury to persons:**

- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have "pinch point" protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the Head or Leg lift section to support concentrated weight such as sitting or standing

## CAUTION

- 1) When lowering any section of the table, make sure that no supplies or any part of yours or your client's body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.
- 2) When lowering the table height, be sure there are no foreign obstacles underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head or leg section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet and electrical components. Evidence of exposure to water or moisture will void your warranty.
- 7) If you have purchased the Flexible Armrest Option, never use the Armrests as handholds or body support mechanisms, especially when setting clients on or off the table.
- 8) Never drop or insert any object into any opening.

## SECTION 2: PRODUCT OVERVIEW

	Everest	Everest Spa	Everest Eclipse
<b>Product Image*</b>			
<b>Available Tops</b>	Flat, Manual Tilt, Pneumatic Tilt, Pneumatic Salon, Electric Tilt, Electric Salon	Flat, Manual Tilt, Pneumatic Tilt, Pneumatic Salon, Electric Tilt, Electric Salon	Flat, Manual Tilt, Electric Tilt, Electric Salon
<b>Top Length</b>	73"	73"	73"
<b>Top Width</b>	28", 30", or 32"	28", 30", or 32"	28", 30", or 32"
<b>Top Cushioning</b>	3" Deluxe Pro-Plush™	3" Deluxe Pro-Plush™	3" Deluxe Pro-Plush™
<b>Top Corner Style</b>	Round	Square	Square
<b>Height Range</b>	Manual: 24" - 40" Pneumatic Tilt: 25" - 41" Flat: 24" - 40"	Manual: 24" - 40" Pneumatic Salon: 26" - 42" Electric Tilt: 26" - 42"	Manual: 24" - 40" Pneumatic Salon: 26" - 42" Electric Tilt: 26" - 42"
<b>Base</b>	Heavy duty steel	Heavy duty steel with hardwood Maple trim	Hardwood cabinet base with 2 drawers
<b>Base Size</b>	44" L x 22" W	47.5" L x 20.5" W	56.25" L x 26.5" W x 15.75" H
<b>Base Finish</b>	White	Natural Maple (Other colors optional)	Black (Other colors optional)
<b>Lift System</b>	Heavy duty 3 stage lift column	Heavy duty 3 stage lift column	Heavy duty 3 stage lift column
<b>Lift Capacity</b>	600 lbs	600 lbs	600 lbs
<b>Upholstery</b>	Natursoft™	Natursoft™	Natursoft™

\*All tables shown with optional accessories and/or finishes. Measurements may vary due to natural material variations.

## SECTION 3: INSTALLATION

Your Everest table comes completely assembled and ready to use.

Once the table is unpacked, follow the “Moving Your Table” instructions below.

### Moving Your Table



#### CAUTION

Your Everest table is very heavy - extreme caution should be used to avoid injury to yourself, others around you, and your table. DO NOT lift or move without using proper safety procedures/equipment. DO NOT turn the table on its side for any reason. Lift your table by grasping firmly from the bottom of the metal frame underneath the upholstered top. Do not try to lift from the upholstered top.

### Basic Setup Instructions

Once your Everest table is positioned for use, follow these steps:

- Plug the power cord into a grounded outlet that is properly rated for the table.
- Arrange and secure the power cord so it will not create a tripping hazard.
- Place the hand and foot controls in locations where they are safe and accessible.
- Ensure the controls are not in danger of being crushed by the lift or tilt mechanisms.
- Follow the “Leveling Your Table” instructions on the next page.






Be certain to arrange the power cord and foot control so they will not create a tripping hazard, and will not be in the path of moving parts.

### Leveling Your Table

Leveling, or balancing, your table is very important during initial setup and routine maintenance. Your Everest table is equipped with leveling feet that can adjust to uneven floors.


To adjust the leveler feet:

1. Place the table in the position you plan to use it.
2. To determine which foot needs adjusting, attempt to “rock” your table. If there is any movement, the leg that is lifting is the one that will need adjusting.
4. Using your hand, or a wrench if necessary, turn the leveling foot in the needed direction. (Counterclockwise will lower the overall height, clockwise will raise the overall height)

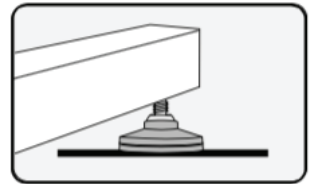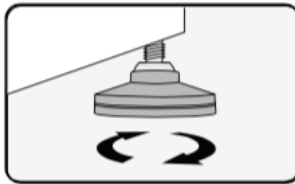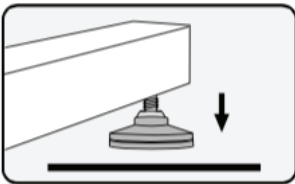

### Table Accessories

All Everest tables come with outlets at both ends, specifically designed for providing a fixed location for FacePillow platforms, salon headrest, and other accessories.

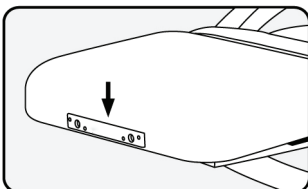


See the Earthlite website for all available accessories that will fit into the table outlets.

# SECTION 4: OPERATING INSTRUCTIONS


## Important Safety Notes



Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts. The described adjustments can be done with a client on the table, but use caution to ensure that the client's weight is supported while positioning the table section(s).

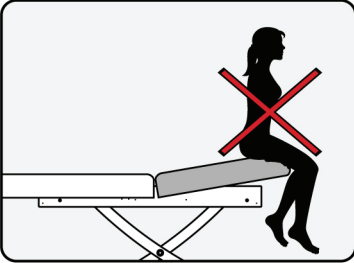

Do not sit or put unnecessary amounts of pressure on the back section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

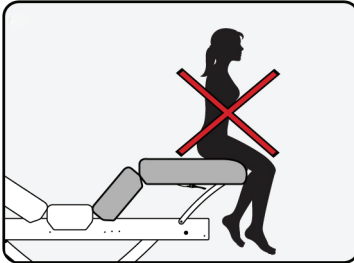

Do not sit or put unnecessary amounts of pressure on the leg section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.



DO NOT store or place anything under the backrest - this can damage the top and/or tilting mechanism and will void your warranty. This area is not designed for storage or as an area to hold accessories and/or massage supplies while performing any type of service.

DO NOT store or place anything between the table and the base - this can damage the table and/or base and will void your warranty. The base of the table is not designed for storage or as an area to hold accessories, including foot pedal.

## Weight & Lift Capacities

**600 lbs.**

The Everest is powered by a heavy-duty 3-stage lift column. The recommended lift capacity for all Everest tables is 600 lbs.

**50 lbs.**

The (optional) back and leg section has a maximum weight capacity of 50 lbs. These sections are not meant to hold full body weight.







# SECTION 4: OPERATING INSTRUCTIONS

## Electrical Specifications

Spec	North America	Europe/Asia
Input Voltage	120 VAC	220 VAC
Input Frequency	60 Hz	50 Hz
Consumption	2.5 Amps, 287.5 Watts	1.3 Amps, 287.5 Watts
Output	32 VDC	32 VDC
Duty Cycle	10%, 2 min/18 min	10%, 2 min/18 min

China


### PLUG DESCRIPTION

NORTH AMERICA	UK / MIDDLE EAST	ASIA	EUROPE
 120V AC	 220V AC	 220V AC	 220V AC


### Key Lockout (ETL Option Only)

If ordered, your table is equipped with a safety lockout. The table will only operate when the magnetic “key” is inserted into the lock. Remove key when table is not in use to prevent unauthorized use and potential injury. Located in head section cabinet.

Note: ETL option is not available for electric tilt and electric salon tops

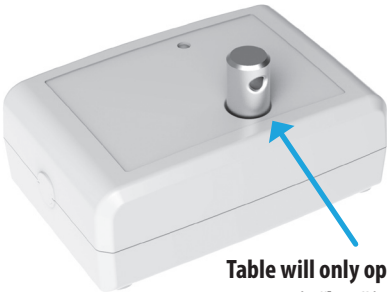

**Table will only operate when magnetic “key” is inserted as shown**

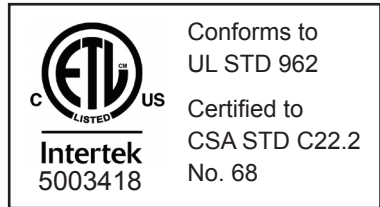


**SAFETY ZONE** - The treatment area must have 3-ft minimum unobstructed clearance around the entire table. Posting a notice is recommended to maintain the “Safety Zone Clearance”. The designated work area around the table should be clearly marked off.

# SECTION 4: OPERATING INSTRUCTIONS

## Foot Control Operation

Your Everest foot control is already connected and ready for use. The foot control allows you to simply raise and lower the table, allowing convenient hands-free adjustments.

### Standard Foot Pedal

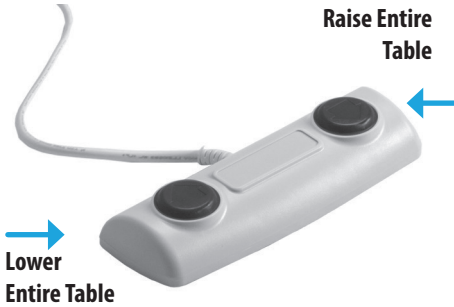

### ETL option




## Hand Control Operation (Electric Tilt/Salon Only)

Your Everest hand control is already connected and ready for use. See below for a description of each button's functionality.




## SECTION 4: OPERATING INSTRUCTIONS

### Pneumatic Back Tilt (Optional on Selected Models)

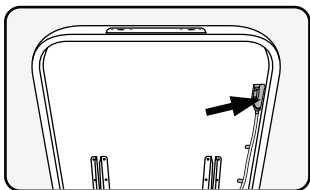


#### Step 1

Locate the pneumatic back tilt handle, which is located on the right side under the back tilt.

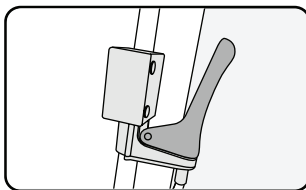


#### Step 2

The handle is set to the locking position.

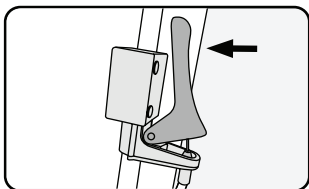

#### Step 3


Squeeze / press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

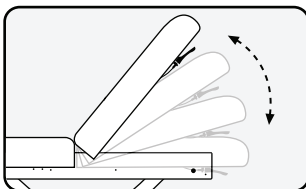

#### Step 4


With the handle squeezed / pressed, move the back tilt to the desired level and release the handle. It will lock in place.

### Pneumatic Leg Section (Optional on Selected Models)

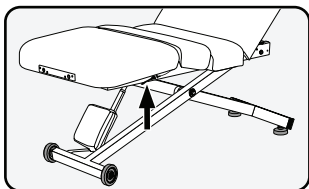


#### Step 1

Locate the pneumatic leg break control, which is located under the front section of the table.

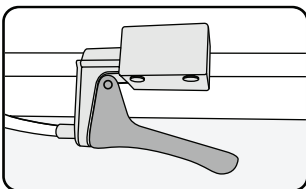


#### Step 2

The handle is set to the locking position.

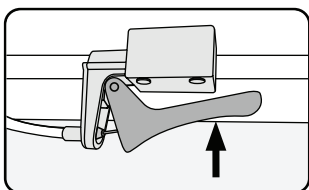

#### Step 3

Squeeze/press the handle to release the pneumatic assist, which allows you to move the back tilt up or down.

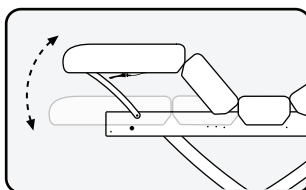


#### Step 4

With the handle squeezed/pressed, move the back tilt to the desired level and release the handle. It will lock in place.

**Note:** the above images are for instructional purposes only - your table/base style may


## SECTION 4: OPERATING INSTRUCTIONS

### Manual Back Tilt (Optional on Selected Models)

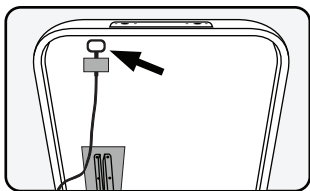


#### Step 1

To tilt the back section upward from its flat position, carefully place both hands underneath the rear edge of the upholstered top of the tilt section, as pictured.

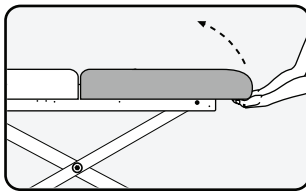


#### Step 2

Lift the tilt section of the table to the desired level. The tilt section will automatically 'click' into position as you lift upward.

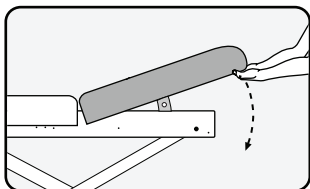

#### Step 3

To return to a flat position, release the locking mechanism by lifting the back section all the way forward past 90°, until the locking mechanism releases.

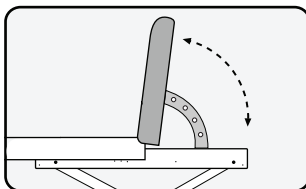

#### Step 4

Once the locking mechanism is released, lower the tilt section until it is flat (1), then lift again to another desired angle (2).

**Note:** the above images are for instructional purposes only - your table/base style may


### Warming Drawer Operation (Optional for Eclipse)

**IMPORTANT:** Please refer to separate Warming Drawer User Manual for safety warnings and instructions for use. If you did not receive one, contact Earthlite Customer Service before use to be sent an electronic copy.


## SECTION 4: OPERATING INSTRUCTIONS

### LED Chroma Lighting Hand Control Operation (Eclipse Only)

With the remote, you can change the color to any one of 16 colors, or use the preset color changing patterns. See below for instructions on how to use the included remote control:

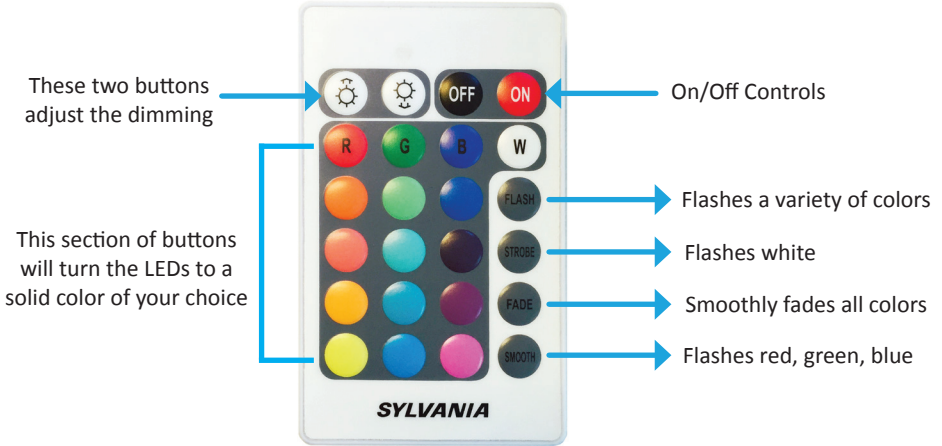

**NOTE:** The LED remote control works with infrared and requires an unobstructed line of sight to the receiver. The LED receiver is located in the center of the table on the underside, by the column. You will need to raise the table a few inches for line of sight.

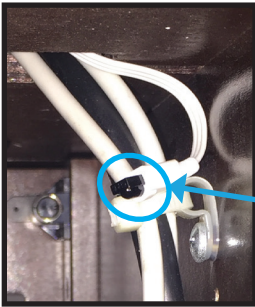

# SECTION 5: CARE & MAINTENANCE

## Cleaning & Ongoing Maintenance



### RISK OF ELECTRIC SHOCK

Always unplug this furnishing from the electrical outlet before cleaning.



DO NOT USE acetone, "Armor All" type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

With proper maintenance, your table is designed to give you years of trouble-free operation. To ensure your table will continue to function properly and keep its appearance, we recommend the following.

Category	Recommendation	Application
<b>Upholstery</b>	Remove day-to-day dirt with a mild soap and warm water solution. For tough stains and to disinfect, use a 15:1 water to bleach solution. Always rinse surface with clean water, and then gently wipe the upholstery. Allow upholstery to air dry before using or storing. Do NOT use Armor All® or citrus-based cleansers. For best results, use only the products suggested above. Alternative cleansing methods may cause serious damage to your table. Such damage would not be covered under the Earthlite warranty program.	<ul style="list-style-type: none"> <li>• Table Bed Surface</li> <li>• Flex-Armrests</li> <li>• Headrest Cushion</li> <li>• Salon Pillow</li> <li>• Bolsters</li> </ul>
<b>Steel Table Frame</b>	The powder coated steel frame can be kept clean with a mild detergent and warm water mix. Periodically remove dust from frame with a dry, soft cloth.	<ul style="list-style-type: none"> <li>• Main Steel Frame and Legs</li> </ul>
<i>Electronic Components</i>	Anti-static electronic aerosol dusters can be used to keep the surfaces of electronic components, power cables, and plugs free of dust.	<ul style="list-style-type: none"> <li>• Foot Controls</li> <li>• Hand Controls</li> <li>• Motors</li> <li>• Power Cables</li> <li>• Plugs</li> </ul>
<b>Lubrication &amp; Mechanical</b>	If necessary, lubricate wheel joints, casters, foot glide levelers, ball joints, and other mechanical parts with a light machine oil. Periodically check table and ensure all bolts remain tightened.	<ul style="list-style-type: none"> <li>• Wheel Joints</li> <li>• Casters</li> <li>• Foot Glide Levelers</li> <li>• Ball Joints, etc.</li> </ul>

# SECTION 5: CARE & MAINTENANCE

## Troubleshooting

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your table, determine possible causes, and find a solution.

In the event your specific situation is not covered in this guide, please contact Earthlite so our technical support department can assist you. Beyond the troubleshooting guide below, do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Earthlite Customer Service first at (800) 872-0560, (760) 599-1112 Int'l.

Issue	Solution
Table does not raise or lower	<ol style="list-style-type: none"><li>1. Ensure the table is plugged in.</li><li>2. If using a dimmer switch, ensure it is turned to full power.</li><li>3. Ensure the outlet in your room is receiving power. This can be done by plugging in a small appliance and turning it on.</li><li>4. Ensure the switch on the attached power strip is turn on.</li><li>5. Check the fuse or circuit breaker for blown fuse or tripped breaker.</li><li>6. Ensure the controls are securely plugged into the junction box.</li></ol>
One-Touch Lever is difficult to move	Check the cable for any kinks or sharp bends from the lever to the gas springs. The cable should have a soft radius in order to allow the inner cable to travel freely in the cable sheath. If the cable is kinked, contact Earthlite.

## Absolute Satisfaction Return Policy

If you are not completely satisfied with your Earthlite portable table, chair, or accessory, you may return it within thirty (30) days for a full refund (excluding shipping & handling charges on the original order). Standard stationary and lift tables may be returned within thirty (30) days for a refund (excluding shipping & handling charges on the original order) less a 20% restocking fee. Custom or upgraded products are non-returnable and non-refundable. Proof of purchase is required to obtain RMA (Return Material Authorization) and all returns must be in like-new condition and in the original packaging. Customers will be responsible for all shipping charges.

## Warranty Information

Earthlite offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty
- Foam and upholstery covering - warranted for three (3) years
- Lift motors, controls, electronics - warranted for two (2) years from the original date of purchase\*
- Accessories and options - warranted for one (1) year from the original date of purchase

This warranty does not cover damage caused by normal wear and tear, misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by cleaners. All warranties are limited to repair or replacement at Earthlite's discretion, shipping not included. Proof of purchase is required to obtain RMA (Return Material Authorization) and customers may be required to return defective items. International customers are responsible for all associated return costs (inbound and outbound), as well as any applicable duties and taxes.

\*If within these two (2) years, there is a defect, malfunction, or failure of the lift motor or controls, Earthlite will replace or repair the defective component at our discretion; shipping not included. Parts are warranted for 90 days after replacement or 2 years from the original date of table purchase, whichever comes later.



# EarthLite®

WORLD'S #1 BRAND IN MASSAGE™

## Contact Us

Earthlite  
990 Joshua Way  
Vista, CA. 92081  
(800) 872-0560  
(760) 599-1112 Int'l  
(866) 599-7374 fax  
info@earthlite.com

Hours of operation:

Monday - Friday  
8:00 a.m. to 5 p.m. PST

Saturday - Sunday  
Closed

Visit us online at: [earthlite.com](http://earthlite.com)



Join us on  
Facebook



Follow us on  
Twitter



Follow us on  
Instagram



Connect with us  
on Linked In