



Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Living Earth Crafts®.

© Copyright 2024
Living Earth Crafts®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Living Earth Crafts.

Living Earth Crafts® is a registered trademark of Earthlite, LLC.

Living Earth Crafts® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Living Earth Crafts® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Living Earth Crafts® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

Introduction

Congratulations on your purchase of the innovative Radiance PT, the industry’s first multi-purpose LED light therapy treatment table. The Radiance PT makes it easy to utilize Celluma®’s FDA cleared LED light therapy into face and body treatment protocols to dramatically enhance guest benefits and experiences. Impressive treatment versatility enables spas to maximize its profitability with a wide range of attended and unattended treatments.

Radiance PT tables are designed for licensed therapists and are intended to be used in a professional spa, therapy, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The Radiance PT table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 597-2155 for international customers. We can also be reached via e-mail at info@livingearthcrafts.com

Table of Contents

Section 1: Safety Information	
Safety Symbols	2
Warnings	2-3
Section 2: Product Overview	
Product Photos	4
Product Specifications	5
Section 3: Installation	
Safety Lockout	6
Initialization/Reset Instructions	7
Leveling Instructions	8
Section 4: Operating Instructions	
Important Safety Notes	9
Hand/Foot Control	10
LED Lighting	11
LED Light Therapy System	12-13
Conforma Lux™ Chest Comfort System	14
Embedded Warmer	15
Section 5: Care & Maintenance	
Care Instructions	16
Troubleshooting	17
Warranty Information	17
Contact Us	back cover

SECTION 1: SAFETY INFORMATION

Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Living Earth Crafts product.

IMPORTANT: The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.




RISK OF ELECTRIC SHOCK, FIRE, AND EXPLOSION

- 1) Always unplug this furnishing from the electrical outlet before servicing.
- 2) Never use this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- 3) Keep the cord away from heated surfaces.
- 4) Do not use outdoors.
- 5) Do not operate where aerosol (spray) projects are being used or where oxygen is being administered.
- 6) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 7) Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.

GROUNDING INSTRUCTIONS:

POWER SUPPLY: (120V AC grounded) Electrical limitations: Do not use extension cord with this product, plug directly into grounded outlet. **This product must be grounded.** If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. This three prong grounded plug will only fit into wall outlet one way. If the plug does not fit fully in the outlet, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

NORTH AMERICA ONLY

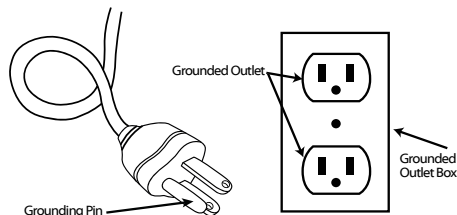

INTERNATIONAL VERSIONS ONLY

This product is for use on a nominal 220V-240V circuit and has a grounding plug (plug type varies by country). Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

SECTION 1: SAFETY INFORMATION

WARNING



To reduce the risk of burns, fire, electric shock, or injury to persons:

- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have "pinch point" protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the Head or Leg lift section to support concentrated weight such as sitting or standing

CAUTION

- 1) When lowering any section of the table, make sure that no supplies or any part of yours or your client's body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.
- 2) When lowering the table height, be sure there are no foreign obstacles underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head or leg section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet and electrical components. Evidence of exposure to water or moisture will void your warranty.
- 7) If you have purchased the Flexible Armrest Option, never use the Armrests as handholds or body support mechanisms, especially when setting clients on or off the table.
- 8) Never drop or insert any object into any opening.

SECTION 2: PRODUCT OVERVIEW

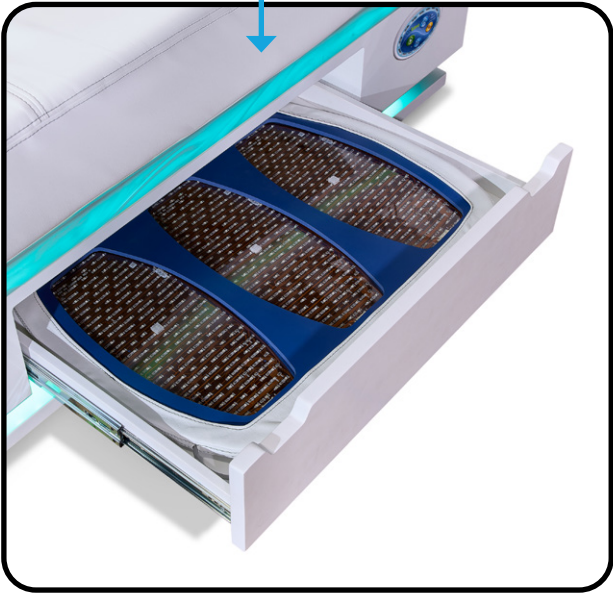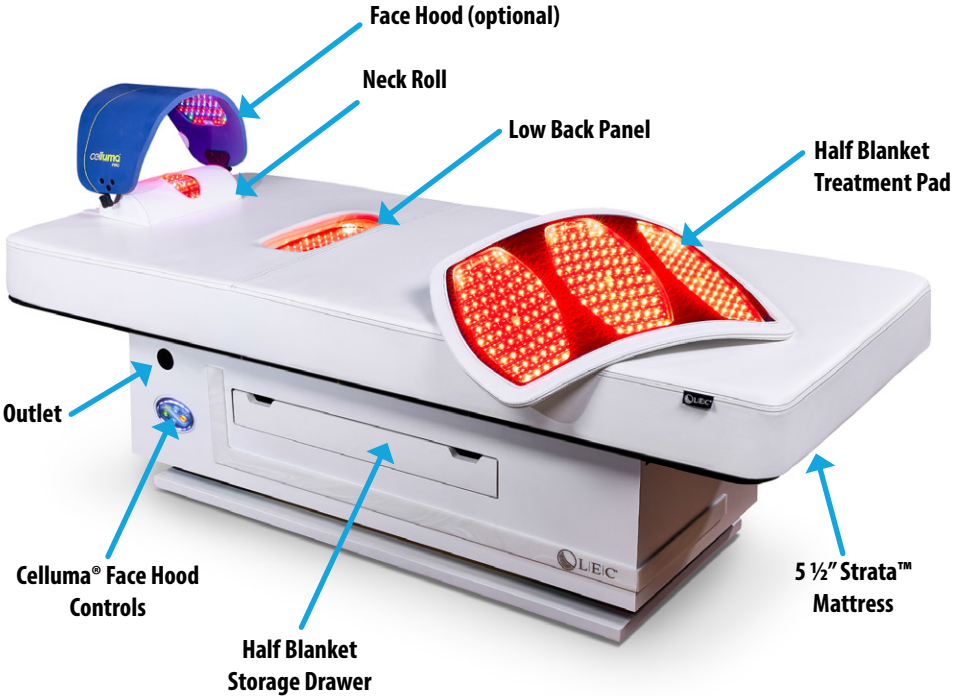

4 *Table shown with optional accessories and/or finishes.
Measurements may vary due to natural material variations.

SECTION 2: PRODUCT OVERVIEW

Product Specifications

Top Length	73" (185 cm)
Top Width	30", 32" (76 cm, 81 cm)
Base Size	48" L x 21.25" W (122 cm L x 54 cm W)
Height Range	22" - 37.5" (52 cm - 91 cm)
Lift Capacity	500 lbs

Features

FEATURE	STANDARD	OPTIONAL
Commercial-grade frame	X	
Fully motorized operation	X	
5 ½" Strata™ mattress	X	
Wireless controls	X	
5 Panel Red Light System	X	
Celluma® Face Hood		X
2 red light storage drawers	X	
LED chromatherapy lighting	X	
Heat at Foot End		X
Mind-Sync™ Restorative Vibrational Wellness		X
ReZen™ Acoustic Resonance Therapy		X
PivotPerfect™ arm rests		X
Conforma Lux™ Adaptive Chest Comfort System		X
Table Wheels		X
6 button foot control		X

SECTION 3: INSTALLATION

Unpacking & Moving Your Table



Your Radiance PT table weighs 535 pounds and requires 5 people to safely unpack and move the table. Extreme caution should be used to avoid injury to yourself, others around you, and your table. DO NOT lift or move without using proper safety procedures/equipment. DO NOT turn the table on its side for any reason.

REFER TO SEPARATE UNPACKING/MOVING INSTRUCTIONS. IF YOU DID NOT RECEIVE UNPACKING / MOVING INSTRUCTIONS, CONTACT LIVING EARTH CRAFTS CUSTOMER SERVICE BEFORE ATTEMPTING TO UNPACK OR MOVE YOUR TABLE.

Safety Lockout

Your table is equipped with a safety lockout. The table will only operate when the magnetic pin is inserted into the "lock". Remove pin when table is not in use to prevent unauthorized use and potential injury. Safety lockout is located in center under table.

NOTE: Pin is included in a plastic bag attached to unpacking/moving instructions and Technical Service Guide for table initialization and reset.

Pin must be inserted as shown below before table can be used.

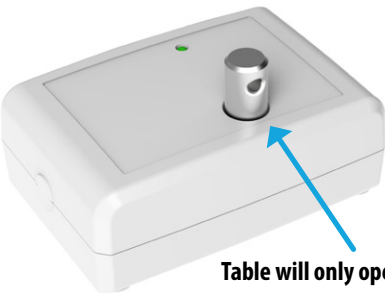


Table will only operate when magnetic pin is inserted as shown

The logo consists of a circle containing the letters "ETL" in a stylized font. Below the circle, the word "LISTED" is written in a smaller font. To the left of the circle is a small "C" and to the right is a small "US". Intertek 5003418	Conforms to UL STD 962
	Certified to CSA STD C22.2
	No. 68

SECTION 3: INSTALLATION

FOLLOW BELOW INSTRUCTIONS BEFORE FIRST USE:

Reset Instructions


NOTE: TO PROPERLY RESET THE SYSTEM ALL OF THE BELOW STEPS NEED TO BE SUCCESSFULLY COMPLETED. IT IS RECOMMENDED THAT A RESET BE DONE ONCE PER WEEK FOR BEST PERFORMANCE OF YOUR TABLE.

Step 1: Raise head and foot sections approximately 1". Raise columns approximately 1" as well.

Step 2: Press and hold Keys 5 and 6 at the same time on hand control (shown in photo below) until columns are at lowest level. Columns will slowly retract until both are completely down. Continue to hold until the table lowers and then raises 1". Reset is complete.

NOTE: IF UNABLE TO COMPLETE RESET, OR COLUMNS ARE MOVING OUT OF SYNC AFTER COMPLETING RESET INSTRUCTIONS, FOLLOW THE STEPS BELOW:

Step 1: If the two columns are too far out of sync, see Step 2 above.

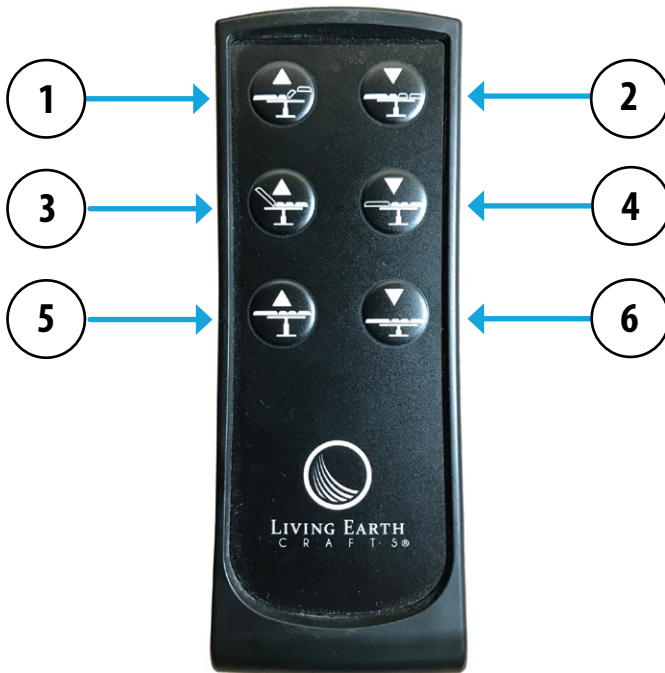

SECTION 3: INSTALLATION

Basic Setup Instructions

Once your Radiance PT table is positioned for use, follow these steps:

- Plug the power cord into a grounded outlet that is properly rated for the table.
- Arrange and secure the power cord so it will not create a tripping hazard.
- Place the hand and foot controls in locations where they are safe and accessible.
- Ensure the controls are not in danger of being crushed by the lift or tilt mechanisms.
- Follow the “Leveling Your Table” instructions below.



Be certain to arrange the power cord and foot control so they will not create a tripping hazard, and will not be in the path of moving parts.

Leveling Your Table

Leveling, or balancing, your table is very important during initial setup and routine maintenance. Your table is equipped with leveling feet that can adjust to uneven floors.

To adjust the leveler feet:

1. Place the table in the position you plan to use it.
2. To determine which foot needs adjusting, attempt to “rock” your table. If there is any movement, the leg that is lifting is the one that will need adjusting.
4. Using your hand, or a wrench if necessary, turn the leveling foot in the needed direction. (Counterclockwise will lower the overall height, clockwise will raise the overall height)




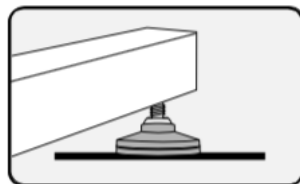


Table Accessories

All tables come with outlets at both ends, specifically designed for providing a fixed location for FacePillow platforms, salon pillows, and other accessories.

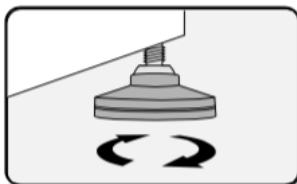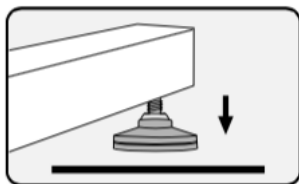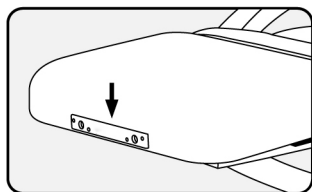

See the Living Earth Crafts website for all available accessories that will fit into the table outlets.

Weight & Lift Capacities

500 lbs.


The Radiance PT table is powered by a 2 heavy-duty lift columns. The recommended lift capacity is 500 lbs.

50 lbs.

The back and leg sections have a maximum weight capacity of 50 lbs. These sections are not meant to hold full body weight.

SECTION 4: OPERATING INSTRUCTIONS

Important Safety Notes

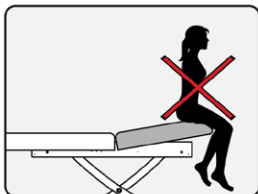

Do not sit or put unnecessary amounts of pressure on the back section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.



Do not sit or put unnecessary amounts of pressure on the leg section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

SEE BELOW PHOTO FOR PINCH POINT ZONES, and read the following warnings carefully regarding safe use of your table.

NOTE: This table does not have pinch point protection.



Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts and pinch points.



DO NOT store or place anything under the backrest or footrest - this can damage the top and/or tilting mechanism and will void your warranty. This area is not designed for storage or as an area to hold accessories and/or massage supplies while performing any type of service.

DO NOT store or place anything between the cabinet and the base - the base of the table is not designed for storage or as an area to hold accessories, including foot pedal. This can damage the table, base, and/or accessories, and will void your warranty.




SECTION 4: OPERATING INSTRUCTIONS

Hand Control Operation

The hand control is already connected and ready for use. See below for a description of each button's functionality.

Note: The wireless reception range is approximately 3 meters.

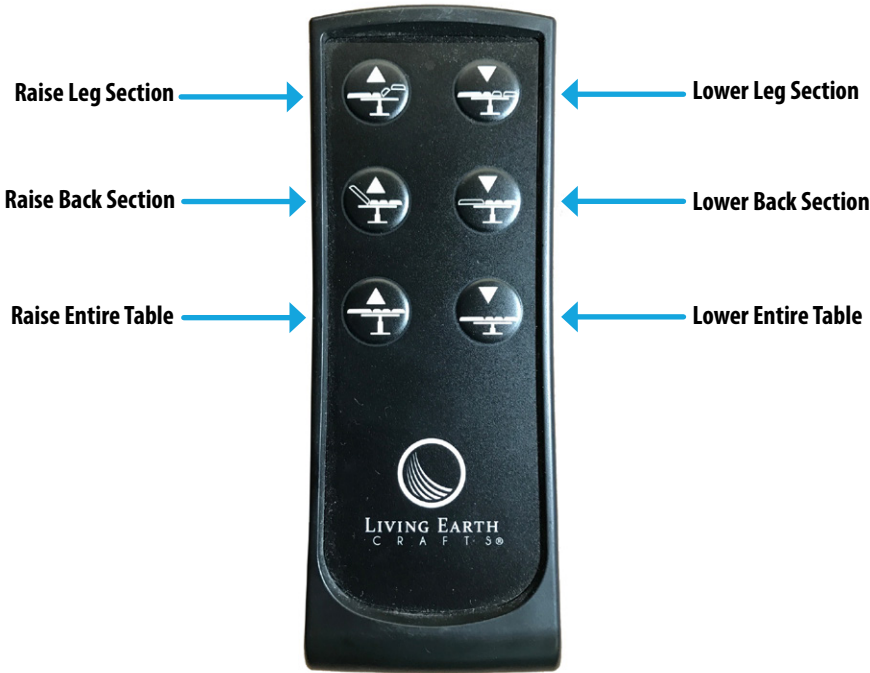

Foot Control Operation


Your foot control is already connected and ready for use. The foot control allows you to simply raise and lower the table, allowing convenient hands-free adjustments.




SECTION 4: OPERATING INSTRUCTIONS

LED Chroma Lighting Hand Control Operation

Your Radiance PT table is equipped with color-changing LED accent lighting, allowing you to match any decor or client preference. With the remote, the user can change the color to any one of 16 colors, or use the preset color changing patterns. See below for instructions on how to use the included remote control:

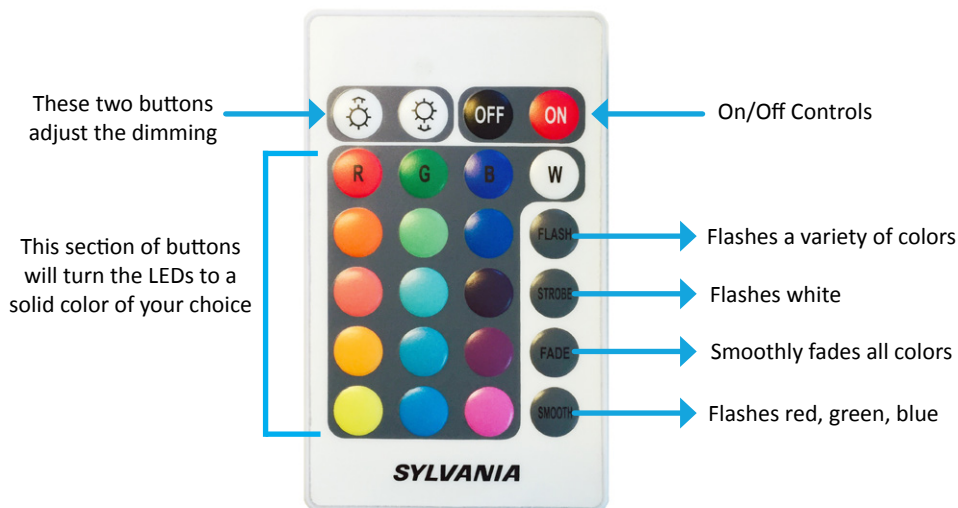

NOTE: The LED remote control works with infrared and requires an unobstructed line of sight to the receiver. The LED receiver can be found in the cabinet to the left of the warming drawer

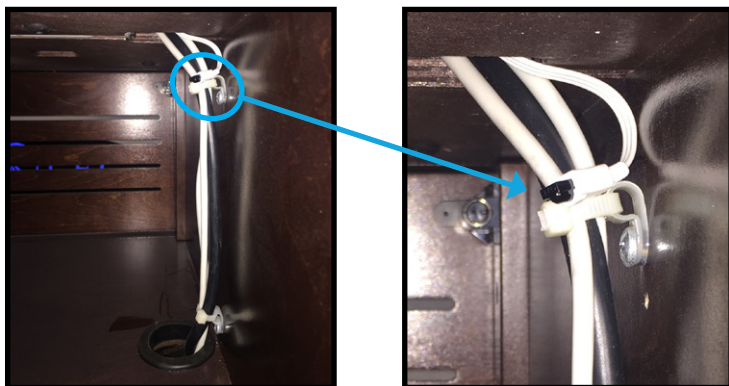

SECTION 4: OPERATING INSTRUCTIONS

LED Light Therapy System

The innovative Radiance PT treatment table incorporates proprietary neck and low back LED light panels, a large flexible “half blanket” treatment pad, as well as an optional curved facial and scalp hood. See below for how to operate the neck, low back, and half blanket treatment panels.



DO NOT put full body weight using a hand, elbow, knee, or other body part on any of the red light panels. Any weight placed on the panels needs to be properly distributed.

Neck, Low Back, and Flexible Blanket Treatment Pads:


To operate the neck, low back, and blanket treatment pads, there are on/off buttons on the side of the table. To operate, simply press the button corresponding to the panel you would like to activate. A light will illuminate when the panel is ON. See below for which button activates each panel.



Neck Roll Panel On/Off

Low Back Panel On/Off

Half Blanket Panel On/Off

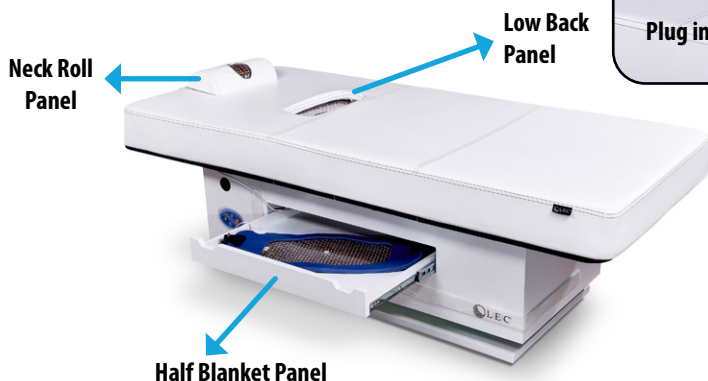

Plug included to cover low back panel when not in use

SECTION 4: OPERATING INSTRUCTIONS

Celluma® Face/Scalp Panel (Optional)

For your Celluma® curved facial and scalp hood, there are five FDA cleared light therapy treatment options that include pain management, inflammation, body contouring, wrinkle and skin repair, and hair restoration. Each incorporates wavelengths of blue, red, and near-infrared light energy simultaneously emitted in proprietary combinations to achieve maximum restorative results.

- FDA cleared, Class II OTC medical device
- Wavelengths: Blue: 465nm / Red: 640nm / Near-Infrared: 880nm
- Dose and Intensity: Varies by treatment. 5.24-7.01 J/cm² at 1" distance
- Recommend using protective eye goggles for facial treatments
- Safe, non-invasive, UV-free, and non-toxic

IMPORTANT: Below are basic instructions on how to operate the Celluma embedded control panel for the optional Face Hood. For full instructions and safety information, refer to the included Celluma® instruction manual.



1. Press the yellow power button to turn on.
2. Select treatment with the select button at the top of the panel.
3. Press the green start button. This starts a 30 min treatment and the panel will power off when complete.
4. Neck Roll, Back Panel, and Half Blanket all default to Pulse mode.
5. The bottom button is the pulse on/off button, a red light will illuminate next to the button to indicate it is on. *(Pulse mode provides the most effective treatment. However if the mode is irritating to the user it can be switched off and is still effective.)*

SECTION 4: OPERATING INSTRUCTIONS

Chest Comfort Operation (Optional)

If your Radiance PT table is equipped with Conforma Lux™ Adaptive Chest Comfort System, read below instructions.



Always use caution when making any position adjustments while client is on the table.





Adjustment Paddle Location

Paddle is located at the head end of the table, as shown above.

To Lower Recess: Press down on the paddle

To Raise Recess: Press up on the paddle

**Different table model shown for illustrative purposes only - paddle location for the Radiance PT is the same as shown.*

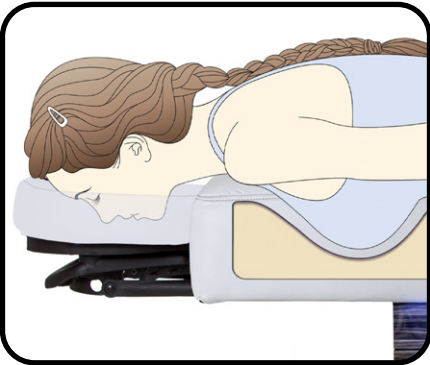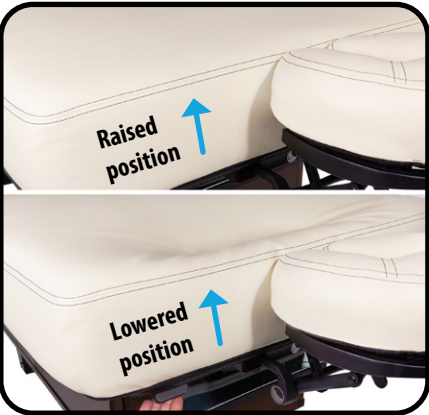

Side View in lowered position

NOTE: If you need to reset your Conforma Lux™ Adaptive Chest Comfort System, simply raise the panel to the highest location to reset.


SECTION 4: OPERATING INSTRUCTIONS

Electrical Specifications

Spec	North America	Europe
Input Voltage	120 VAC	220 VAC
Input Frequency	60 Hz	50 Hz
Input Consumption	2.5A, 300 Watts	1.36A, 300 Watts
Consumption - LED Lights	Additional 0.5 Amps	Additional 0.28 Amps
Consumption - Table Warmer	Additional 0.6 Amps	Additional 0.33 Amps
Consumption - Conforma Lux™ Adaptive Chest Comfort System	Additional 2.5 Amps	Additional 1.38 Amps
Output	12-24 VDC	12-24 VDC

Dual Zone Table Warmer (Optional)

Your table is equipped with an embedded dual zone table warmer, that can be operated by the below control panel. The control panel will be mounted to your table cabinet.

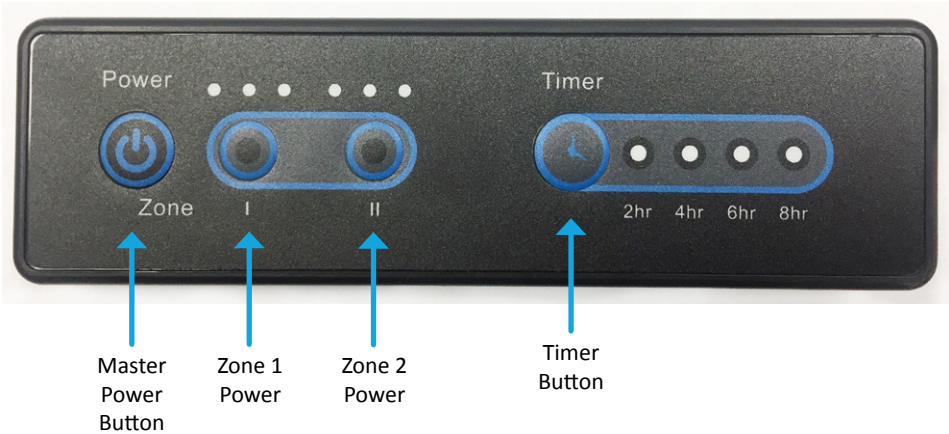

NOTE: Zone 1 heats the back section of the table, and Zone 2 heats the leg section of the table.

Operating Instructions

1. Press master power button - this will turn on zone 1 and 2 to low heat, with a 2 hour timer.
2. Press zone 1 or zone 2 power buttons to choose different heat levels.
3. Press the timer button to choose different timer intervals.
4. Press the master power button again to turn off the table warmer. (Note: when you turn the table warmer back on again, it will remember your last settings.)



To ensure safety and optimal performance, it is recommended to perform a monthly inspection of the table warmer to ensure proper placement of the pad (no bunching), and look for any signs of wear or damage to electrical connections.

SECTION 5: CARE & MAINTENANCE

Cleaning & Ongoing Maintenance



RISK OF ELECTRIC SHOCK

Always unplug this furnishing from the electrical outlet before cleaning.



DO NOT USE acetone, “Armor All” type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

With proper maintenance, your table is designed to give you years of trouble-free operation. To ensure your table will continue to function properly and keep its appearance, we recommend the following.

Category	Care Recommendation
Upholstery	<p>Remove day-to-day dirt and smudges with a mild soap and water solution. For tough stains and to disinfect, use a (100 to 1) water to bleach solution. Spray a damp cloth with the solution to clean.</p> <p>After each cleaning, gently wipe the upholstery with a damp cloth and clean water, then allow upholstery to air dry before using or storing.</p> <p>For more difficult stains, please contact your dealer or please call us directly for guidance. For best results, use only the products suggested above.</p>
LED Panels	<p>Clean after each use with a mild soap and water solution. To disinfect, use a (100 to 1) water to bleach solution. Spray a damp cloth with the solution to clean. After cleaning, gently wipe with a damp cloth and clean water, then allow to air dry before using or storing.</p>
Laminate	<p>To clean the surface, use a damp cloth or sponge and a mild soap or detergent. Rinse thoroughly with warm water and wipe dry.</p>
Electronics	<p>Anti-static electronic aerosol dusters can be used to keep the surfaces of electronic components, power cables, and plugs free of dust.</p>
Lubrication & Mechanical	<p>The actuator or mechanical lift is maintenance free. Do not lubricate. Be careful to avoid application of sticky substances to the mechanism.</p>
Leveling Feet	<p>Regularly check to ensure the table is level and stable. See page 5 for leveling instructions.</p>

**To keep your table working properly, it is recommended that a system reset be done once a week

SECTION 5: CARE & MAINTENANCE

Troubleshooting

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your table, determine possible causes, and find a solution.

In the event your specific situation is not covered in this guide, please contact Living Earth Crafts so our technical support department can assist you. Beyond the troubleshooting guide below, do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Living Earth Crafts Customer Service first. Living Earth Crafts Customer Service: (800) 358-8292, (760) 597-2155 Int'l

Issue	Solution
Table does not raise or lower	<ol style="list-style-type: none">1. Ensure the table is plugged in.2. If using a dimmer switch, ensure it is turned to full power.3. Ensure the outlet in your room is receiving power. This can be done by plugging in a small appliance and turning it on.4. Ensure the switch on the attached power strip is turned on.5. Check the fuse or circuit breaker for blown fuse or tripped breaker.6. Ensure the controls are securely plugged into the junction box.
Columns are out of sync	See reset instructions on page 7.

Warranty Information

Living Earth Crafts (LEC) offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty*
- Foam and upholstery covering - warranted for three (3) years
- Motors, controls, electronics - warranted for two (2) years, including parts and labor
- Accessories and options - warranted for one (1) year, unless otherwise noted

LEC may repair or replace defective components at our discretion. LEC will return the warranted product, standard shipping pre-paid, to customers in the continental US. International customers are responsible for any applicable duties and taxes, as well as all shipping charges inbound and outbound. Normal wear and tear, and costs associated with maintenance procedures, and damage caused by accident, misuse, improper maintenance, harsh operating environments or neglect (which includes tears in the upholstery and damage caused by harsh chemical-based cleaners) are not covered under this warranty. All warranties are limited to repair or replacement, at LEC's discretion, unless otherwise stated.

*Lifetime warranty applies to the original owner, as long as you own your table for 5 years after the model is discontinued, or 5 years for private label tables.



Contact Us

Living Earth Crafts
990 Joshua Way
Vista, CA. 92081
(800) 358-8292
(760) 597-2155 Int'l
(866) 449-8872 fax
info@livingearthcrafts.com

Hours of operation:

Monday - Friday
8:00 a.m. to 5 p.m. PST

Saturday - Sunday
Closed

Visit us online at: livingearthcrafts.com



Join us on
Facebook



Follow us
on X



Follow us on
Instagram



Connect with us
on Linked In