



**EarthLite**  
WORLD'S #1 BRAND IN MASSAGE™

## **USER MANUAL**

### **Apex Chiropractic Table**



#### Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Earthlite®.

© Copyright 2019  
Earthlite®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Earthlite.

Earthlite® is a registered trademark of Earthlite, LLC.

Earthlite® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Earthlite® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Earthlite® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

# Introduction

Congratulations on your purchase of the Apex treatment table. The table has been designed for maximum comfort and to meet the specific positioning needs of your clients.

Earthlite tables are designed for licensed therapists and are intended to be used in a professional spa, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the product for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 599-1112 for international customers. We can also be reached via e-mail at [info@earthlite.com](mailto:info@earthlite.com).

## Table of Contents

<b>Section 1: Safety Information</b>	
Safety Symbols .....	2
Warnings .....	2-3
<b>Section 2: Product Overview</b>	
Product Photos .....	4
Product Specifications .....	5
<b>Section 3: Installation</b>	
Moving Instructions .....	6
Leveling Instructions .....	6
Weight/Lift Capacities .....	6
Electrical Specifications .....	6
<b>Section 4: Operating Instructions</b>	
Foot Control Operation .....	7
Head Rest Adjustment .....	7
Manual Height Adjustment .....	7
<b>Section 5: Care &amp; Maintenance</b>	
Care Instructions .....	8
Troubleshooting .....	8
Warranty Information .....	9
Contact Us .....	back cover

# SECTION 1: SAFETY INFORMATION

## Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

**IMPORTANT:** The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

## IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.




### RISK OF ELECTRIC SHOCK, FIRE, AND EXPLOSION

- 1) Always unplug this furnishing from the electrical outlet before servicing.
- 2) Never use this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- 3) Keep the cord away from heated surfaces.
- 4) Do not use outdoors.
- 5) Do not operate where aerosol (spray) projects are being used or where oxygen is being administered.
- 6) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 7) Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.

## GROUNDING INSTRUCTIONS:

**POWER SUPPLY:** (120V AC grounded) Electrical limitations: Do not use extension cord with this product, plug directly into grounded outlet. **This product must be grounded.** If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. This three prong grounded plug will only fit into wall outlet one way. If the plug does not fit fully in the outlet, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

### NORTH AMERICA ONLY

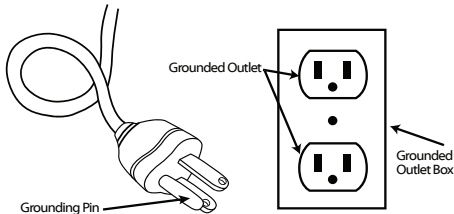

### INTERNATIONAL VERSIONS ONLY

This product is for use on a nominal 220V-240V circuit and has a grounding plug (plug type varies by country). Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

# SECTION 1: SAFETY INFORMATION

## WARNING

**To reduce the risk of burns, fire, electric shock, or injury to persons:**


- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have "pinch point" protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the head rest section to support concentrated weight such as sitting or standing

## CAUTION


- 1) When lowering any section of the table, make sure that no supplies or any part of yours or your client's body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.
- 2) When lowering the table height, be sure there are no foreign obstacles underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head rest section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet and electrical components. Evidence of exposure to water or moisture will void your warranty.
- 7) Never drop or insert any object into any opening.

# SECTION 2: PRODUCT OVERVIEW

## Apex Stationary

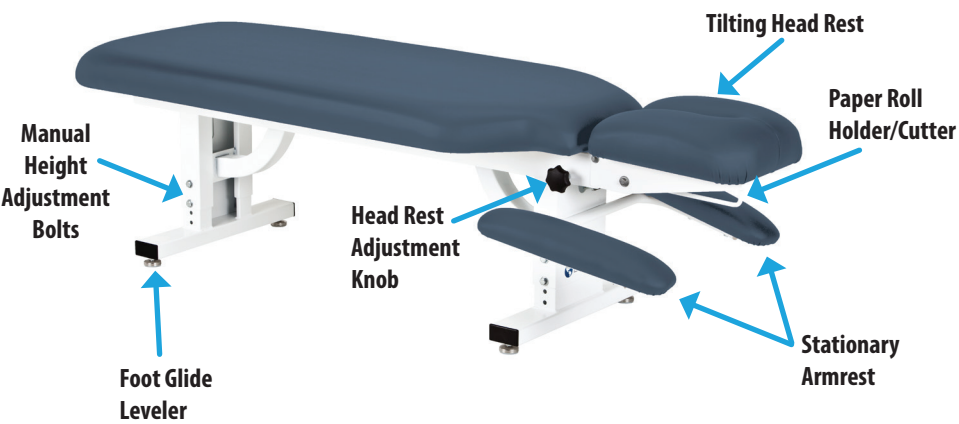

## Apex Lift

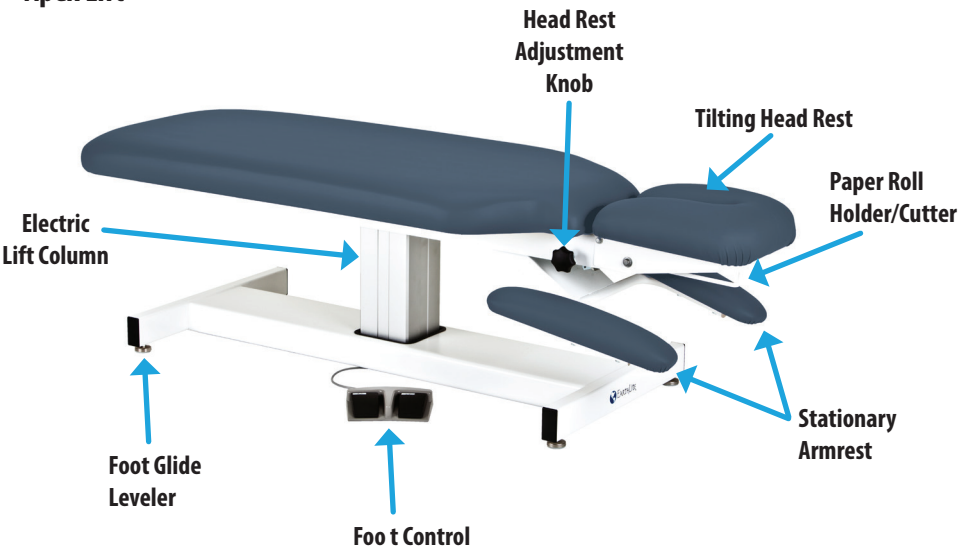

Measurements may vary due to natural material variations.

## SECTION 2: PRODUCT OVERVIEW

### Product Specifications

Table Width	24"
Table Length	72"
Height Range	22.5" - 31.5"
Weight/Lift Capacity	400 pounds

### Included Features

Feature	Apex Stationary	Apex Lift
Heavy Duty Steel Frame	X	X
Tilting Head Rest	X	X
Stationary Armrest	X	X
Seamless Natursoft™ Upholstery	X	X
3" High Density Foam Cushioning	X	X
Paper Roll Holder and Cutter	X	X
Manual Height Adjustment	X	
Electric Lift Column Height Adjustment		X
Heavy Duty Foot Control		X

# SECTION 3: INSTALLATION

Your Apex table comes completely assembled and ready to use. Once the table is unpacked, follow the “Moving Your Table” instructions below.

## Moving Your Table



Your table is very heavy - extreme caution should be used to avoid injury to yourself, others around you, and your table. DO NOT lift or move without using proper safety procedures/equipment.




Lift your table by grasping firmly from the bottom of the frame underneath the upholstered top. Do not try to lift from the upholstered top.

## Leveling Your Table

Leveling, or balancing, your table is very important during initial setup and routine maintenance. Your table is equipped with leveling feet that can adjust to uneven floors.

To adjust the leveler feet:

1. Place the table in the position you plan to use it.
2. To determine which foot needs adjusting, attempt to “rock” your table. If there is any movement, the leg that is lifting is the one that will need adjusting.
4. Using your hand, or a wrench if necessary, turn the leveling foot in the needed direction. (Counterclockwise will lower the overall height, clockwise will raise the overall height)

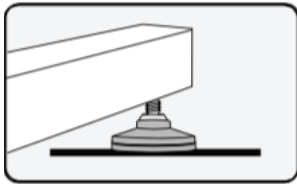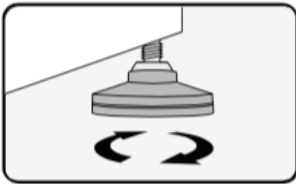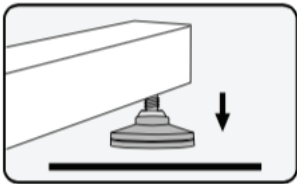

## Weight & Lift Capacities

**400 lbs.**

The Apex recommended weight capacity is 400 pounds

## Electrical Specifications (Apex Lift Only)

Input Voltage	120 VAC
Input Frequency	60 Hz
Consumption	2.5 Amps, 300 Watts max
Output	32 VDC
Duty Cycle	10%, 2 min/18 min



## SECTION 4: OPERATING INSTRUCTIONS

### Important Safety Notes

#### WARNING

Always use extreme caution when making any height adjustments to your table. Ensure all body parts and objects are clear from all moving parts.

#### CAUTION

DO NOT store or place anything between the table and the base - this can damage the table and/or base and will void your warranty. The base of the table is not designed for storage or as an area to hold accessories, including foot pedal.

### Foot Control Operation (Apex Lift Only)

Your hand control is already connected and ready for use. See below for a description of each button's functionality.



### Head Rest Adjustment

**Step 1:** Supporting the head rest with one hand, turn and hold the head rest adjustment knob with your other hand.

**Step 2:** Lift or lower the head rest to the desired position, and release the adjustment knob to lock into position.




### Manual Height Adjustment

**Required Tools:** ½" wrench and socket wrench

**Step 1:** Use wrench to loosen bolt while using socket wrench to hold nut on the other side.

**Step 2:** Remove bolt completely and adjust table to desired height.

**Step 3:** Reinstall bolt and nut, tighten securely.

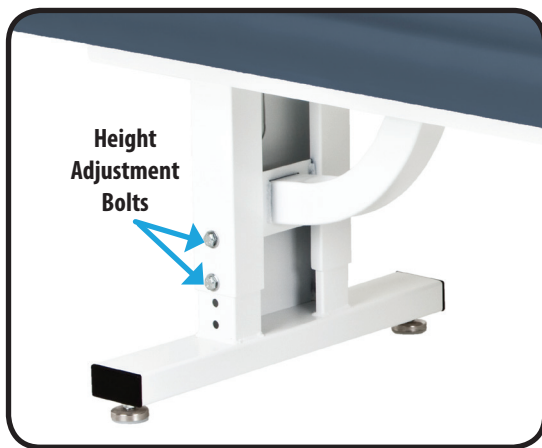

# SECTION 5: CARE & MAINTENANCE

## Cleaning & Ongoing Maintenance



### RISK OF ELECTRIC SHOCK

Always unplug this furnishing from the electrical outlet before cleaning.



DO NOT USE acetone, “Armor All” type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

With proper maintenance, your table is designed to give you years of trouble-free operation. To ensure your table will continue to function properly and keep its appearance, we recommend the following.

Category	Care Recommendation
Upholstery	Remove day-to-day dirt and smudges with a mild soap and water solution. For tough stains and to disinfect, use a (100 to 1) water to bleach solution. Spray a damp cloth with the solution to clean. After each cleaning, gently wipe the upholstery with a damp cloth and clean water, then allow upholstery to air dry before using or storing. For more difficult stains, please contact your dealer or please call us directly for guidance. For best results, use only the products suggested above.
Lubrication & Mechanical	The actuator or mechanical lift is maintenance free. Do not lubricate. Be careful to avoid application of sticky substances to the mechanism.
Leveling Feet	Regularly check to ensure the table is level and stable. See page 6 for leveling instructions.

## Troubleshooting

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your table, determine possible causes, and find a solution.

In the event your specific situation is not covered in this guide, please contact Earthlite so our technical support department can assist you. Beyond the troubleshooting guide below, do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Earthlite Customer Service first at (800) 872-0560, (760) 599-1112 Int'l.

Issue	Solution
Table does not raise or lower	<ol style="list-style-type: none"><li>1. Ensure the table is plugged in.</li><li>2. If using a dimmer switch, ensure it is turned to full power.</li><li>3. Ensure the outlet in your room is receiving power. This can be done by plugging in a small appliance and turning it on.</li><li>4. Check the fuse or circuit breaker for blown fuse or tripped breaker.</li><li>5. Ensure the controls are securely plugged into the junction box.</li></ol>
Table is wobbling or unstable	See “Leveling Your Table” instructions on page 6.

## SECTION 5: CARE & MAINTENANCE

---

### **Absolute Satisfaction Return Policy**

If you are not completely satisfied with your Earthlite portable table, chair, or accessory, you may return it within thirty (30) days for a full refund (excluding shipping & handling charges on the original order). Standard stationary and lift tables may be returned within thirty (30) days for a refund (excluding shipping & handling charges on the original order) less a 20% restocking fee. Custom or upgraded products are non-returnable and non-refundable. Proof of purchase is required to obtain RMA (Return Material Authorization) and all returns must be in like-new condition and in the original packaging. Customers will be responsible for all shipping charges.

### **Warranty Information**

Earthlite offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty
- Foam and upholstery covering - warranted for three (3) years
- Lift motors, controls, electronics - warranted for two (2) years from the original date of purchase\*
- Accessories and options - warranted for one (1) year from the original date of purchase

This warranty does not cover damage caused by normal wear and tear, misuse, accidents, or neglect, which includes tears in the upholstery and damage caused by cleaners. All warranties are limited to repair or replacement at Earthlite's discretion, shipping not included. Proof of purchase is required to obtain RMA (Return Material Authorization) and customers may be required to return defective items. International customers are responsible for all associated return costs (inbound and outbound), as well as any applicable duties and taxes.

\*If within these two (2) years, there is a defect, malfunction, or failure of the lift motor or controls, Earthlite will replace or repair the defective component at our discretion; shipping not included. Parts are warranted for 90 days after replacement or 2 years from the original date of table purchase, whichever comes later.



# EarthLite®

WORLD'S #1 BRAND IN MASSAGE™

## Contact Us

Earthlite  
990 Joshua Way  
Vista, CA. 92081  
(800) 872-0560  
(760) 599-1112 Int'l  
(760) 477-6071 fax  
[info@earthlite.com](mailto:info@earthlite.com)

Hours of operation:

Monday - Friday  
8:00 a.m. to 5 p.m. PST

Saturday - Sunday  
Closed

Visit us online at: [earthlite.com](http://earthlite.com)



Join us on  
Facebook



Follow us on  
Twitter



Connect with us  
on Linked In



**self-certified**

Made in USA  
of U.S. and  
imported parts