


**Nuage™ Table Collection**

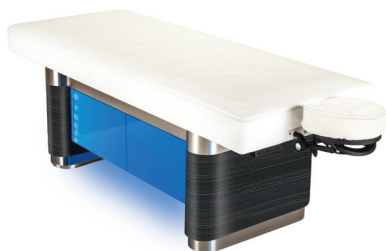

**Nuage™ Designer Cabinetry Series**



**Nuage™ Vector**



**Nuage™ Vector +Studio**



**Nuage™ Pedestal**



**Nuage™ Pedestal +Studio**

© Copyright 2019  
Living Earth Crafts®

All rights reserved. This document or any portion thereof may not be reproduced in any way, including photocopying or translation to another language without the prior written permission from Living Earth Crafts.

Living Earth Crafts® is a registered trademark of Earthlite, LLC.

#### Legal Notice

The information provided in this document is intended for informational purposes only and is subject to change without notice. The information provided in this document should not be construed as a commitment by Living Earth Crafts®.

Living Earth Crafts® assumes no responsibility for the accuracy or suitability of the information contained in this document. This document may contain technical inaccuracies or errors. Living Earth Crafts® makes no expressed or implied warranties of any kind concerning the information contained in this document, including, but not limited to, the implied warranties of merchantability and fitness for purpose.

Living Earth Crafts® shall not be liable for incidental or consequential damages in connection with or arising out of the product, performance, or use of this document and the information and material it describes.

# Introduction

Congratulations on your purchase of a table from the Nuage™ Collection. All Nuage™ tables are designed with advanced technology to bring you the quietest, smoothest, all-electric spa treatment tables in the market today. Each Nuage™ table is designed from the ground up for amazing comfort, ultra-reliability, and minimal maintenance in the most active spa environments.

Nuage™ tables are designed for licensed therapists and are intended to be used in a professional spa, massage studio, or similar commercial environment. While no special training is needed to operate the table, for safety reasons it is important that the instructions contained in this manual are reviewed in detail before the first use. This manual should be read in its entirety by each professional who will be operating the table to ensure the safety of both operator and client. This manual should be saved for future reference.

The Nuage™ table should not be used in any other manner than described in this manual. Please read, understand, and follow all safety warnings, labels, and instructions. Misuse, use of unnecessary force, or using the table for unintended or unauthorized procedures can result in injury to operator, client, and/or equipment.

Detailed instructions for use and maintenance can be found in this user manual. If you have further questions, please contact us at (800) 872-0560 for US customers, or (760) 597-2155 for international customers. We can also be reached via e-mail at [info@livingearthcrafts.com](mailto:info@livingearthcrafts.com)

## Table of Contents

<b>Section 1: Safety Information</b>	
Safety Symbols .....	2
Warnings & Grounding Instructions .....	2-3
<b>Section 2: Product Overview</b>	
Product Photos & Specifications .....	4-5
<b>Section 3: Installation</b>	
Moving Your Table .....	6
Basic Setup .....	6
Leveling Instructions .....	7
Weight/Lift Capacities .....	7
<b>Section 4: Operating Instructions</b>	
Important Safety Notes .....	8
Electrical Specifications .....	8
Hand/Foot Control .....	9
+Studio Options .....	10-14
<b>Section 5: Care &amp; Maintenance</b>	
Care Instructions .....	15
Troubleshooting .....	16-17
Warranty Information .....	17
Contact Us .....	back cover

# SECTION 1: SAFETY INFORMATION

## Safety Symbols

Familiarize yourself with the following Safety & Warning symbols. They are designed to prevent damage and injury to you, your clients, and your new Earthlite product.

**IMPORTANT:** The important safety instructions and warnings in this manual cannot cover all possible problems and conditions that can occur. Use common sense and caution when installing, operating, or maintaining this appliance.



Indicates an imminently hazardous situation which could result in serious or fatal injury.



Indicates a potentially hazardous situation which could result in serious injury.



Indicates a potentially hazardous situation which could result in equipment damage and/or injury.

## IMPORTANT SAFETY INSTRUCTIONS - FOR COMMERCIAL USE ONLY

Read all instructions before using this product.




### RISK OF ELECTRIC SHOCK, FIRE, AND EXPLOSION

- 1) Always unplug this furnishing from the electrical outlet before servicing.
- 2) Never use this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the furnishing to a service center for examination and repair.
- 3) Keep the cord away from heated surfaces.
- 4) Do not use outdoors.
- 5) Do not operate where aerosol (spray) projects are being used or where oxygen is being administered.
- 6) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 7) Connect this furnishing to a properly grounded outlet only. See Grounding Instructions.

## GROUNDING INSTRUCTIONS:

**POWER SUPPLY:** (120V AC grounded) Electrical limitations: Do not use extension cord with this product, plug directly into grounded outlet. **This product must be grounded.** If it should malfunction or breakdown, grounding provides a path of least resistance for electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. This three prong grounded plug will only fit into wall outlet one way. If the plug does not fit fully in the outlet, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

### NORTH AMERICA ONLY

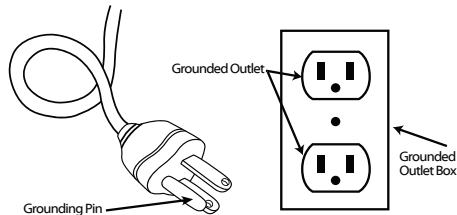

### INTERNATIONAL VERSIONS ONLY

This product is for use on a nominal 220V-240V circuit and has a grounding plug (plug type varies by country). Make sure that the product is connected to an outlet having the same configuration as the plug. No adapter should be used with this product.

# SECTION 1: SAFETY INFORMATION

## WARNING

**To reduce the risk of burns, fire, electric shock, or injury to persons:**



- 1) Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
- 2) Do not operate this equipment unless properly trained.
- 3) Install this furnishing in treatment area not accessible to the public.
- 4) Close supervision is necessary when this furnishing is used by, or near children, invalids, or disabled persons.
- 5) This product does not have "pinch point" protection. Only operate when no possibility of any portion of the body could be caught and injured while moving the table top upward/downward, or activating the back/foot tilt.
- 6) Do not use this furnishing to support video equipment such as televisions or computer monitors.
- 7) Never attempt to use the Head or Leg lift section to support concentrated weight such as sitting or standing

## CAUTION

1) When lowering any section of the table, make sure that no supplies or any part of yours or your client's body is between the top of your table and its base. This could result in injury, or could damage your table as well as the item. It is recommended that you always return your table top to the lowest flat position to avoid leaving the base exposed.


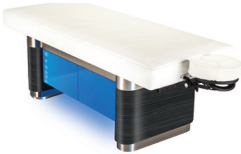

- 2) When lowering the table height, be sure there are no foreign obstacles underneath your table that could obstruct it from lowering completely. This could damage your table.
- 3) Never set anything under the table frame to avoid collision if lowered.
- 4) Never operate your table on an uneven surface. Adjust the leveling feet so that your table is stable.
- 5) Never raise the head or leg section with a client in the prone position (face down.)
- 6) Never use or store your table in a wet or very humid environment. Water and excessive moisture can damage the wood cabinet and electrical components. Evidence of exposure to water or moisture will void your warranty.
- 7) If you have purchased the Flexible Armrest Option, never use the Armrests as handholds or body support mechanisms, especially when setting clients on or off the table.
- 8) Never drop or insert any object into any opening.

# SECTION 2: PRODUCT OVERVIEW

	Nuage Designer Cabinetry	Nuage Pedestal
Product Image*		
Available Tops	Flat, Electric Salon	Flat, Electric Salon
Top Length	73"	73"
Top Width	30" or 32"	30" or 32"
Top Cushioning	Strata 4" Memory Foam	Strata 4" Memory Foam
Top Corner Style	Round	Round
Top Degree of Tilt (Electric Salon Only)	Back Tilt: 0° - 70° Leg Tilt: 0° - 43°	Back Tilt: 0° - 70° Leg Tilt: 0° - 43°
Height Range	25" - 37"	25" - 37"
Cord Exit Location	Head end of table	Head end of table
Diagnostic Panel Location	Centered inside cabinet	Centered underneath top
Base	3 Cabinetry Options	Pedestal Base
Base/Cabinet Measurements	Traditional, Contemporary 52.5" x 23.25" x 21" Cuvee 54.5" x 23.75" x 21"	51.5" x 22.25" x 21"
Base Construction	Cross braced steel frame Hardwood Maple cabinet	Cross braced steel frame Hardwood Maple base
Included Accessories		
Programmable Hand Control	X	X
Foot Control	X	X
PowerPort Outlets	X	X
Salon Style Neck Roll	X	X
Embedded Table Warmer		
Smart Tablet		
Strata FacePillow w/Platform		
LED/Chromatherapy Lighting		
SO Sound		
Stereo Speaker		
Aromatherapy System		
Warming Drawer <sup>1</sup>		

\*all tables shown with optional accessories and/or finishes. Measurements may vary due to natural material variations.  
1 - see Warming Drawer User Manual for instructions for use.

## SECTION 2: PRODUCT OVERVIEW

Nuage Pedestal +Studio	Nuage Vector	Nuage Vector +Studio
		
Electric Salon	Electric Salon	Electric Salon
73"	73"	73"
30" or 32"	30" or 32"	30" or 32"
Strata 3" Memory Foam (+ 1" embedded warmer)	Strata 3" Memory Foam (+ 1" embedded warmer)	Strata 3" Memory Foam (+ 1" embedded warmer)
Square	Square	Square
Back Tilt: 0° - 70° Leg Tilt: 0° - 43°	Back Tilt: 0° - 70° Leg Tilt: 0° - 43°	Back Tilt: 0° - 70° Leg Tilt: 0° - 43°
25" - 37"	25" - 37"	25" - 37"
Head end of table	Head end of table	Head end of table
Underneath back tilt section	Centered underneath top	Centered underneath top
Pedestal Base	Dual Pedestal Base	Dual Pedestal Base
51.5" x 22.25" x 21"	51.5" x 22.25" x 21"	51.5" x 22.25" x 21"
Cross braced steel frame w/laminate	Cross braced steel frame w/laminate	Cross braced steel frame w/laminate
X	X	X
X	X	X
X	X	X
X	X	X
X		X
		X
		X
X	X	X
		X
		X
		X
X		

## SECTION 3: INSTALLATION

Your Nuage™ table comes completely assembled and ready to use.

Ensure you read and follow the unpacking instructions included with your shipment. If you did not receive unpacking instructions, please contact L·E·C customer service before unpacking your table.

Once the table is unpacked, follow the “Moving Your Table” instructions below.

### Moving Your Table



Your Nuage table is very heavy - extreme caution should be used to avoid injury to yourself, others around you, and your table. DO NOT lift or move without using proper safety procedures/equipment. DO NOT turn the table on its side for any reason.


- To avoid damage to your Nuage™ table, it is very important that it only be lifted from certain positions on the table (table edges marked with **ARROWS** below)
- You may need to plug in your table and raise the table, the back section, and/or the leg section for easier access to the base for lifting purposes.
- Lift your table by grasping firmly from the bottom of the frame. Do not try to lift from the fins on the Cuvee cabinet or the cabinet doors.



Nuage Cabinetry Models



Nuage Pedestal Models

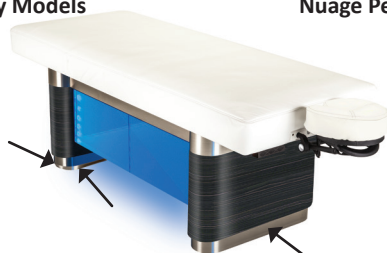

Nuage Vector Models

### Basic Setup Instructions

Once your Nuage™ table is positioned for use, follow these steps:

- Plug the power cord into a grounded outlet that is properly rated for the table.
- Arrange and secure the power cord so it will not create a tripping hazard.
- Place the hand and foot controls in locations where they are safe and accessible.
- Ensure the controls are not in danger of being crushed by the lift or tilt mechanisms.
- Follow the “Leveling Your Table” instructions on the next page.






## SECTION 3: INSTALLATION

### Leveling Your Table

Leveling, or balancing, your table is very important during initial setup and routine maintenance. Your Nuage™ table is equipped with leveling feet that can adjust to uneven floors.


To adjust the leveler feet:

1. Place the table in the position you plan to use it.
2. To determine which foot needs adjusting, attempt to “rock” your table. If there is any movement, the leg that is lifting is the one that will need adjusting.
3. Raise table height approximately 6 inches.
4. Using your hand, or a wrench if necessary, turn the leveling foot in the needed direction. (Counterclockwise will lower the overall height, clockwise will raise the overall height)

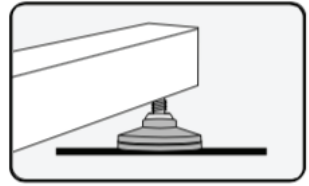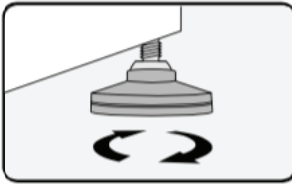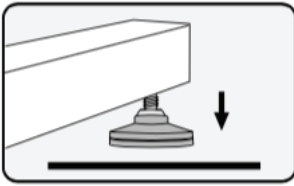

### Table Accessories

All Nuage™ tables come with outlets at both ends, specifically designed for providing a fixed location for FacePillow platforms, salon pillows, and other accessories.

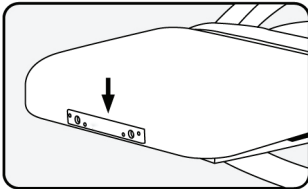

To learn about all available accessories, visit the Living Earth Crafts Website, or call Living Earth Crafts customer service to request a catalog.

### Weight & Lift Capacities

**550 lbs.**

The Nuage table is powered by a 2 heavy-duty lift columns. The recommended lift capacity is 550 lbs.

**50 lbs.**

The back and leg sections have a maximum weight capacity of 50 lbs. These sections are not meant to hold full body weight.

### Warming Drawer (Optional)


**IMPORTANT:** Please refer to separate Warming Drawer User Manual for safety warnings and instructions for use. If you did not receive one, contact LEC Customer Service before use to be sent an electronic copy.

# SECTION 4: OPERATING INSTRUCTIONS


## Important Safety Notes



Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts. The described adjustments can be done with a client on the table, but use caution to ensure that the client’s weight is supported while positioning the table section(s).

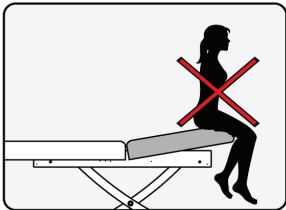

Do not sit or put unnecessary amounts of pressure on the back section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.

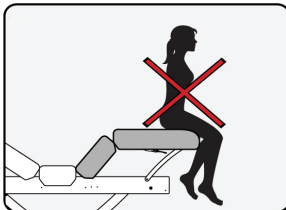





Do not sit or put unnecessary amounts of pressure on the leg section of the table when it is in tilt position of any level. This can damage or destroy the tilt mechanism and cause harm or bodily injury to persons. The tilt mechanism is intended solely to be used when a client is properly seated or laying on the table top. Damage caused in this manner may void the warranty.



DO NOT store or place anything under the backrest - this can damage the top and/or tilting mechanism and will void your warranty. This area is not designed for storage or as an area to hold accessories and/or massage supplies while performing any type of service.

DO NOT store or place anything between the table and the base - this can damage the table and/or base and will void your warranty. The base of the table is not designed for storage or as an area to hold accessories, including foot pedal.

## Electrical Specifications

China <b>PLUG DESCRIPTION</b>			
NORTH AMERICA	UK / MIDDLE EAST	ASIA	EUROPE
 120V AC	 220V AC	 220V AC	 220V AC


\*Other plugs also available for various countries

Spec	North America	Europe/Asia
Input Voltage	120 VAC	220 VAC
Input Frequency	60 Hz	50 Hz
Consumption	7.7 Amps, 925 Watts	4.2 Amps, 925 Watts
Output	12-24 VDC	12-24 VDC
Duty Cycle	10%, 2 min/18 min	10%, 2 min/18 min
Power Strip Rating	12 Amp Max	15 Amp Max

## SECTION 4: OPERATING INSTRUCTIONS

### Hand Control Operation

Your Nuage™ hand control is already connected and ready for use. See below for descriptions of each button's functionality, including 4 convenient memory positions.

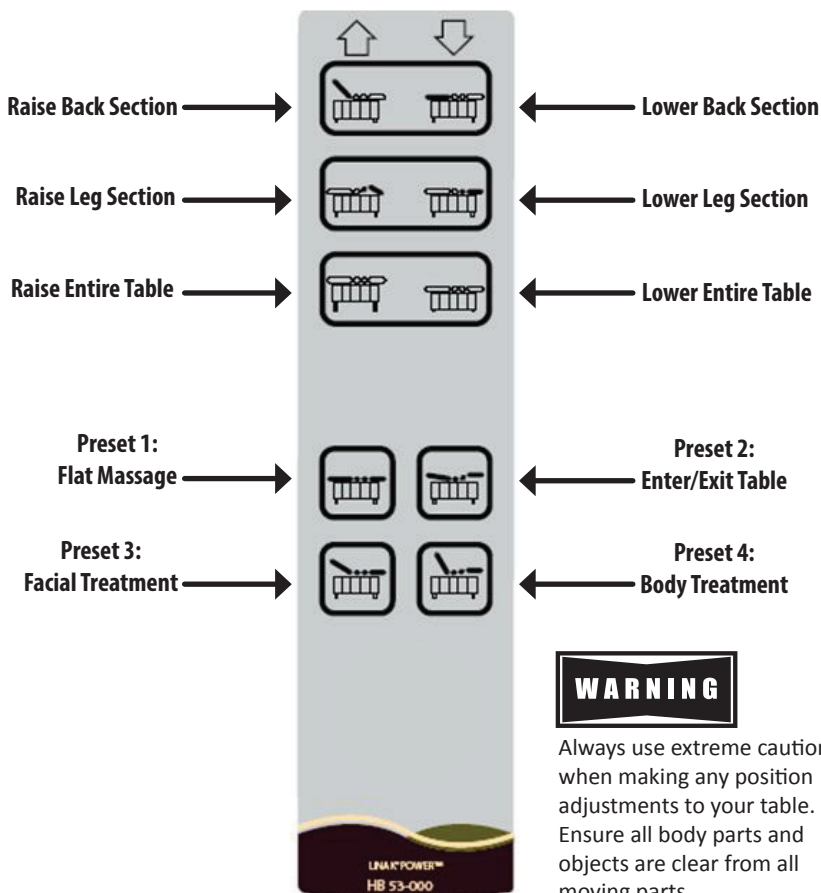

### WARNING

Always use extreme caution when making any position adjustments to your table. Ensure all body parts and objects are clear from all moving parts.

### Foot Control Operation


Your Nuage™ foot control is already connected and ready for use. The foot control allows you to simply raise and lower the table, allowing convenient hands-free adjustments.



# SECTION 4: OPERATING INSTRUCTIONS


## LED Chroma Lighting Hand Control Operation (+Studio Only)

Your +Studio table is equipped with color-changing LED accent lighting, allowing you to match any decor or client preference. With the remote, the user can change the color to any one of 16 colors, or use the preset color changing patterns. See below for instructions on how to use the included remote control:

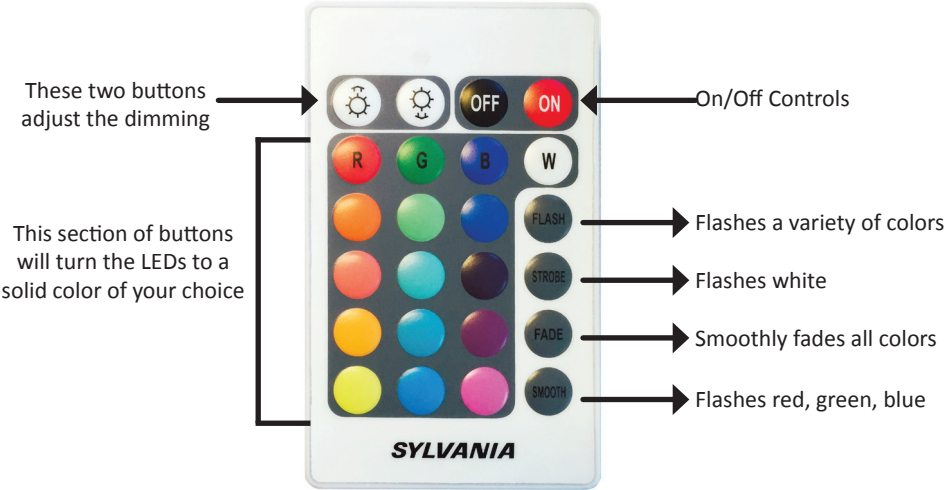

## Dual Zone Table Warmer (+Studio Only)

Your +Studio table is equipped with an embedded dual zone table warmer, that can be operated by the below control panel. The control panel will be mounted to your table cabinet.

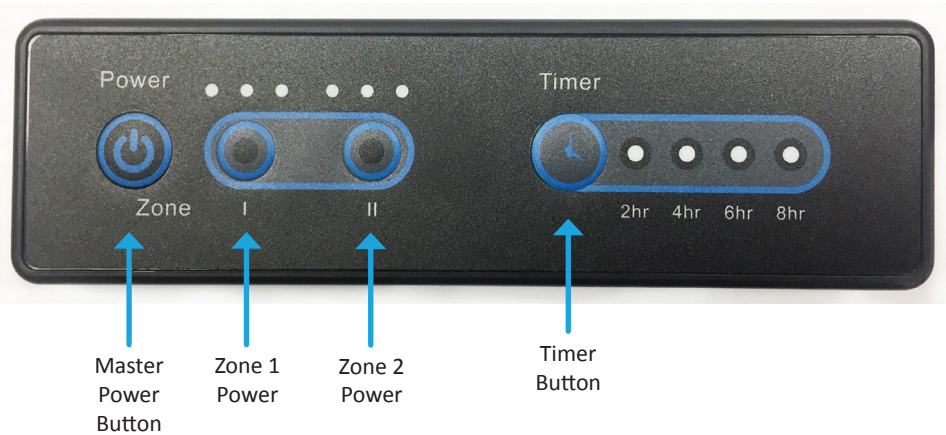

**NOTE:** Zone 1 heats the back section of the table, and Zone 2 heats the leg section of the table.

### Operating Instructions

1. Press master power button - this will turn on zone 1 and 2 to low heat, with a 2 hour timer.
2. Press zone 1 or zone 2 power buttons to choose different heat levels.
3. Press the timer button to choose different timer intervals.
4. Press the master power button again to turn off the table warmer. (Note: when you turn the table warmer back on again, it will remember your last settings.)


## SECTION 4: OPERATING INSTRUCTIONS

### Tablet Instructions (+Studio only)

1. Click on the "Video" icon



2. Click on the video you would like to play

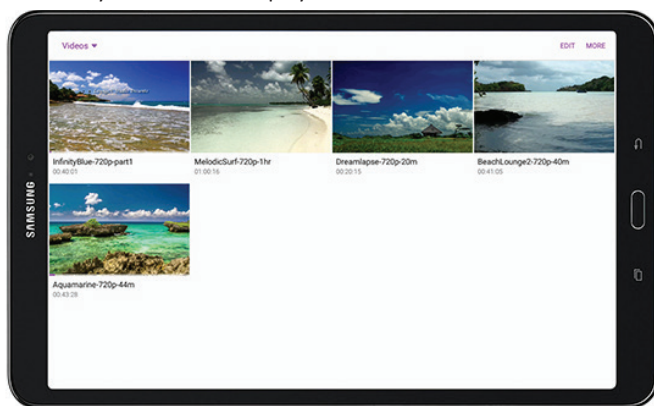

Power    Volume Up/Down



Switch to  
fullscreen  
mode



Pause/Play

# SECTION 4: OPERATING INSTRUCTIONS

## Tablet Instructions (+Studio only)



Pull knob to adjust  
tilt of tablet if needed





## SECTION 4: OPERATING INSTRUCTIONS

### So Sound Overview (Nuage Vector +Studio Only)

So Sound is designed to deliver the healing frequencies of sound and music, tactilely, so you feel the sound and/or music resonating through your body and brain simultaneously to hearing it; thus 'reminding' the body of it's natural state of harmony. This sensory integrative process efficiently and effectively restores a deeper sense of mental, emotional, and physical health and well-being.

### Operating the So Sound System

Your tablet comes pre-connected to the So Sound system, so when you play videos (see instructions on previous page), it will automatically begin to resonate through the So Sound transducers.




## SECTION 4: OPERATING INSTRUCTIONS

### Jawbone Jambox Speaker (Nuage Vector +Studio Only)

The integrated Jawbone Jambox speaker comes pre-connected to the tablet and So Sound system, so when you play the videos (see instructions on page 11), it will automatically begin to play through the speaker.

Playing the music simultaneously through both the So Sound System and the Jawbone speaker gives a more comprehensive sensory integration experience. Speakers are designed to deliver music through the air, while the So Sound System is designed to deliver music through mass. In other words, the frequency ranges that we hear are different from the frequency ranges we feel. By combining the two we create a fully immersive sound healing environment.

The image below shows the controls on your Jambox speaker - the only functions needing adjustment will be the volume controls, using the “+” and “-” buttons.

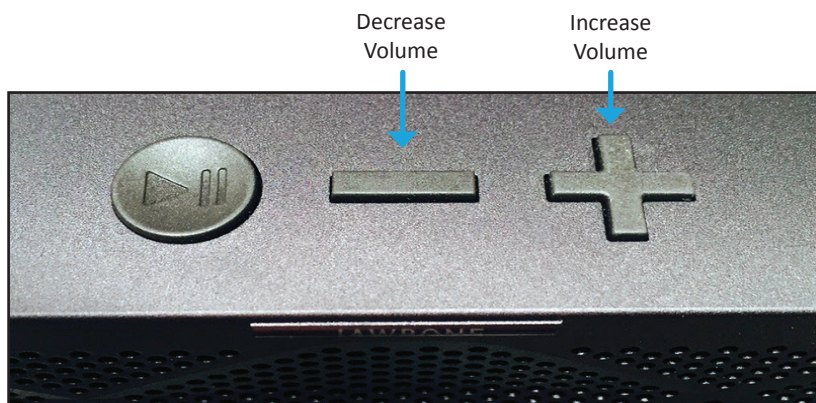

NOTE: Play/Pause/Stop of the videos will need to be controlled through the tablet

### Balancing Your Audio System (Nuage Vector +Studio Only)

The intent of this exercise is to find the optimum balance of sound and resonance

1. Have an individual lay on the table as if they were receiving treatment
2. Lower the resonance to the lowest possible level
3. Set the volume on the Jambox speaker to a medium level
4. Begin playing the videos on the tablet
5. Slowly turn the volume on the Jambox speaker to a comfortable level
6. Adjust the resonance level until it feels good to the individual on the table

NOTE: Once the resonance level is set, it does not need to be further adjusted on the So Sound amplifier. Control both the volume of the speaker and resonance level by changing the volume on the tablet.



## SECTION 5: CARE & MAINTENANCE

### Cleaning & Ongoing Maintenance



#### RISK OF ELECTRIC SHOCK

Always unplug this furnishing from the electrical outlet before cleaning.



DO NOT USE acetone, “Armor All” type cleaners, citrus-based cleaners, or industrial cleaners. These can seriously damage the upholstery. Such damage would not be covered under warranty.

Your Nuage™ table is relatively maintenance free, as the lifting mechanism is enclosed and sealed. However, to ensure your table will continue to function properly and keep its appearance, we recommend the following.

Vinyl	<p>After each treatment on your table, clean thoroughly, removing all residue. This and other residues may cause the vinyl to dry out and become brittle.</p> <p>We recommend a mild detergent and warm water mix, or environmentally friendly disinfectant cleaners, such as Viraguard®. Apply with cloth or spray directly onto the upholstery. Ensure surface is wiped clean of residue.</p> <p>Use caution when attempting to remove stains as you could damage your vinyl. Any stains should be treated immediately with a solution of 1 part alcohol to 10 parts water. Restore the luster to the spot with a light application of spray furniture wax.</p>
Laminate	<p>Never use abrasive cleaners. A mild liquid detergent and warm water will clean soiled surfaces. To avoid discoloration, cleaners containing petroleum distillates must not remain on surfaces.</p>
Wood	<p>To maintain the wood finish and retouch scratches, use a high quality furniture wax.</p>
Electronics	<p>Anti-static electronic aerosol dusters can be used to keep the surfaces of electronic components, power cables, and plugs free of dust.</p>
Lubrication & Mechanical	<p>If necessary, lubricate wheel joints, casters, foot glide levelers, ball joints, and other mechanical parts with a light machine oil. Periodically check and ensure all visible bolts remain tightened.</p>
Leveling Feet	<p>Regularly check to ensure the table is level and stable. See page 7 for leveling instructions.</p>

# SECTION 5: CARE & MAINTENANCE

## Troubleshooting

This helpful troubleshooting guide will provide you with information about how to recognize a problem with your table, determine possible causes, and find a solution.

In the event your specific situation is not covered in this guide, please contact Living Earth Crafts so our technical support department can assist you. Beyond the troubleshooting guide below, do not attempt to repair the table yourself. This can void your warranty.

Unauthorized service will not be covered. Please be sure to contact Living Earth Crafts Customer Service first at (800) 358-8292, (760) 597-2155 Int'l.


Issue	Solution
Table does not raise or lower	<ol style="list-style-type: none"><li>1. Ensure the table is plugged into a grounded and dedicated outlet.</li><li>2. If using a dimmer switch, ensure it is turned to full power.</li><li>3. Ensure the outlet in your room is receiving power. This can be done by plugging in a small appliance and turning it on.</li><li>4. Ensure the switch on the attached power strip is turn on.</li><li>5. Check the fuse or circuit breaker for blown fuse or tripped breaker.</li><li>6. Ensure the controls are securely plugged into the junction box.</li></ol>
Table does not operate in accordance with functions on hand control	<ol style="list-style-type: none"><li>1. Ensure the controls are securely plugged into the junction box.</li><li>2. Perform a table reset (see instructions below).</li></ol>

## SECTION 5: CARE & MAINTENANCE

### Table Reset

Follow these steps to perform a reset of your table - you will be using your table's diagnostic panel to cycle the control box and actuators. This can only be accessed through the cabinet

1. Locate your table's diagnostic panel (as indicated by the red arrows)
2. Press and hold the "down" button on the diagnostic panel, continuing to hold until the table lowers to lowest point.
3. Press and hold the "up" button on the diagnostic panel, continuing to hold until the table raises to highest point, and continue to hold the "up" button until table moves slightly down, then back up again. This signals reset is complete.

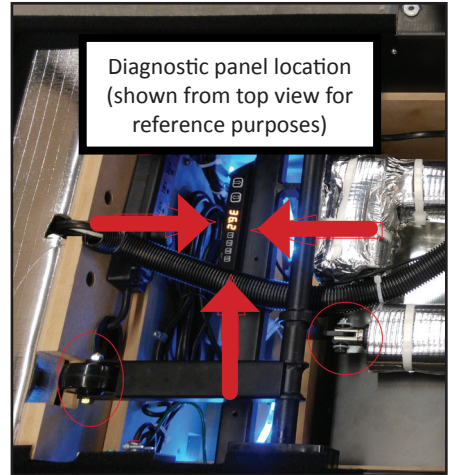

### Warranty Information

Living Earth Crafts (L·E·C) offers an industry best warranty against defects in materials, design, and workmanship, to the original owner, as follows:

- Frame - lifetime warranty\*
- Foam and upholstery covering - warranted for three (3) years
- Motors, controls, electronics - warranted for two (2) years, including parts and labor
- Accessories and options - warranted for one (1) year, unless otherwise noted

L·E·C may repair or replace defective components at our discretion. L·E·C will return the warranted product, standard shipping pre-paid, to customers in the continental US. International customers are responsible for any applicable duties and taxes, as well as all shipping charges inbound and outbound. Normal wear and tear, and costs associated with maintenance procedures, and damage caused by accident, misuse, improper maintenance, harsh operating environments or neglect (which includes tears in the upholstery and damage caused by harsh chemical-based cleaners) are not covered under this warranty. All warranties are limited to repair or replacement, at L·E·C's discretion, unless otherwise stated.

\*Lifetime warranty applies to the original owner, as long as you own your table for 5 years after the model is discontinued, or 5 years for private label tables.



## Contact Us

Living Earth Crafts  
990 Joshua Way  
Vista, CA 92081 USA  
(800) 358-8292  
(760) 597-2155 Int'l  
(866) 449-8872 fax  
[info@livingearthcrafts.com](mailto:info@livingearthcrafts.com)

Hours of operation:

Monday - Friday  
8:00 a.m. to 5 p.m. PST

Saturday - Sunday  
Closed

Visit us online at: [livingearthcrafts.com](http://livingearthcrafts.com)



Join us on  
Facebook



Follow us on  
Twitter



Follow us on  
Instagram



Connect with us  
on Linked In

