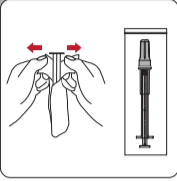
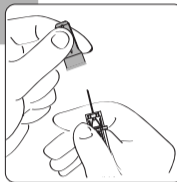
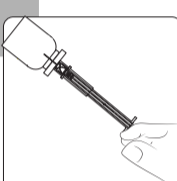
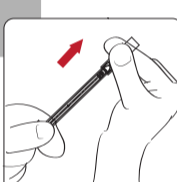
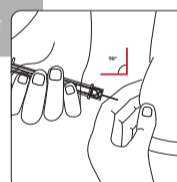
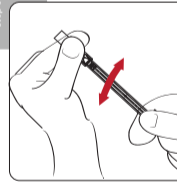
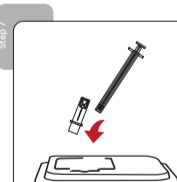


## Instructions for Use - Safe-T-Sheath™ Safety Tuberculin Syringe (Twist Lock)

### INDICATIONS FOR USE

The Safe-T-Sheath Safety Tuberculin Syringe (Twist Lock) is intended to inject medication into the patient. The unique Twist Lock safety shield reduces the risk of needle-stick injury.

- Step 1**  **Peel off the blister pack and remove syringe.**
- Step 2**  **Remove the needle cap.**
- Step 3**  **Draw up medication according to established protocol.**
- Step 4**  **Slide the safety shield over the needle to transport the syringe, this is not permanently locked.**
- Step 5**  **Pull the safety shield back to original position and perform injection.**
- Step 6**  **After injection slide the safety shield over the needle until you hear an audible click. Twist the safety shield in either direction to ensure the needle is permanently locked.**
- Step 7**  **Dispose of the device into an approved sharps container.**

### CAUTIONS

- Immediately use the device after breaking blister package seal.
- In the event the packaging is damaged, or the seal is broken, do not use the product.
- Store the device in a cool, dry environment away from direct sunlight, high heat, and high humidity.
- The device is for single use, DO NOT REUSE.

105 Solana Road- Suite B  
Ponte Vedra Beach  
FL 32092, USA  
E: info@metiermedical.com  
www.metiermedical.com

Local Law restricts this device for sale or use unless approved by a physician.