

DISCOVER
ITS UNIQUE
FEATURES

05

SWISS
knowhow in color technology

SHADE
TECHNOLOGY
DOUBLE CONCISTENCY
SURFACE GLOSS
EFFECT SHADES

inspir 
direct

What about a composite system combining simplicity,
predictability and high esthetics?
inspiro® actually presents an unparalleled blend of features to serve
and please all dentists and direct bonding lovers.

Let's discover them together!

FEATURE #1:

Shade integration and reliability

inspiro has a unique shade guide system which is both simple and effective to select color in a straight forward two steps approach.

If the combined dentin & enamel shades selected do match the patient's natural teeth no surprise, you'll get the color right!





Practically, shade samples are being used first separately; dentin is placed next to the collar of the reference tooth to choose the appropriate dentin chroma and then, enamel is selected simply visually according to its tone and brightness. Finally, shades samples are being combined with a drop of glycerine in-between them to allow optimal light transmission and shade rendering.

Let's get it
confirmed with a

Clinical example

Several class III
restorations have to
be replaced, following
a home bleaching
procedure.

Shade selection can
be done rapidly and
efficiently in 2 simple
steps!






For large cavities, the use of a silicone index helps to control palatal enamel profile and serves as visual reference for the mesio-distal teeth diameter and incisal edge level.



inspiro effect shades allow to obtain individualized, internal color characterization if needed.

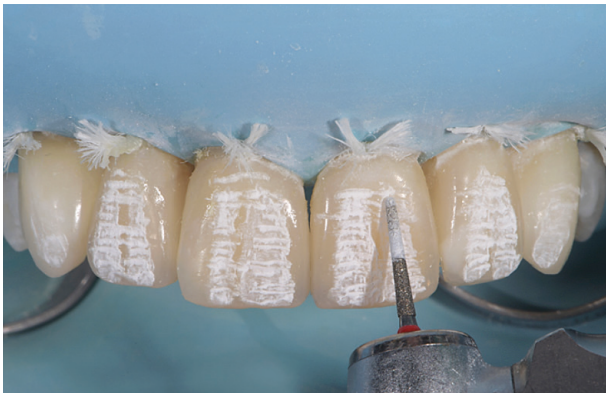

Dentistry by
Dr. Didier Distochi

Completed class III restorations with canine & premolar form corrections and partial veneering, proximal anatomy enhancement of incisors to obtain a more pleasing smile configuration.



Dentistry by

Dr. Didier Dietschi

Do not hesitate to visit our webpage to learn more about all the product features and advantages the inspiro® system can bring to your practice and dentistry.

- www.edelweissdr.com -

